

INSTRUCTION MANUAL

# Basic Buddy® Simulator

LF03693



**5 Year Warranty**

*Nasco*  
HEALTHCARE

# About the Simulator


The Basic Buddy® CPR Simulator is the most realistic, economical manikin available for the training of CPR skills. Features include sanitary airway system, head tilt, realistic chest rise, child and adult compression resistance, palpable landmarks, and a lightweight and compact design. Five-year warranty.

## List of Components

- Base
- Chest
- Head
- Aluminum Screw Post
- Lung Installation Tool
- Package of 10 Lung Bags
- Adult Compression Block

## SET UP

Your Basic Buddy® Simulator is shipped disassembled. Simply follow these easy steps to assemble.

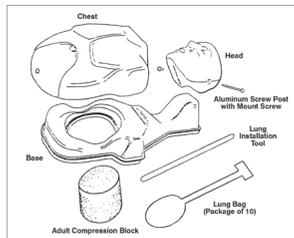

## ATTACHING THE COVER TO THE BASE

Place the chest on top of the base and lock it into place at the three points indicated. **(See Figure 1.)**

## ATTACHING THE HEAD TO THE BASE


Place the head over the neck area of the base and insert the aluminum screw post through the corresponding holes in the head and the base. **(See Figure 2.)** Now insert the mounting holes in the end of the aluminum screw post and tighten. **(See Figure 3.)**



# Set Up

## INSTALLING THE LUNG BAG

Insert the lung installation tool into the lung bag. **(See Figure 4.)** Then insert the bag and tool together into the manikin's mouth while feeding it through the head down to the chest. **(See Figure 5.)** Continuing feeding the tool and the lung bag into the chest cavity until the tool stops against the inside of the chest cavity. **(See Figure 6.)** Remove the lung tool. Make sure the airway lies directly beneath the airway cutoff ridge of the head. Fold the barrier flaps of the lung bag over the face. **(See Figure 7.)** Your Basic Buddy® is now set up for child CPR simulation.

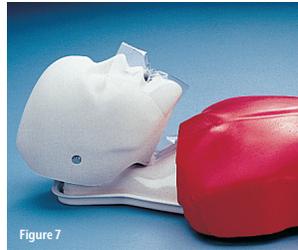

## ADULT CPR SET UP

To set up your Basic Buddy® for adult CPR simulation, simply roll the manikin on its side and insert the adult compression block. **(See Figure 8.)** Removing the adult compression block will change your manikin back to the child compression resistance.



## CLEANING

### Heads:

Heads can be cleaned by any of the following methods:

- Wash heads with soap and water. Then soak 10 minutes in a bleach/water solution (one cup of bleach mixed with one gallon of water),
- OR

- Sanitize with alcohol wipes.

### Bodies:

The bodies can be cleaned by any of the following methods:

- A bleach/water solution (one cup of bleach with one gallon of water),
- OR

- Sanitize with alcohol wipes or any other sanitizing agents.\*

\*Manikin bodies are foam based and are not waterproof. They should not be submerged in water

## MAINTENANCE

The old lung bag should be removed whenever a new student uses the Basic Buddy®. Gently pull the lung bag out of the chest and mouth through the neck opening to avoid contamination. Insert a new lung for each new student.

If you have purchased either the LF03694, Basic Buddy® 5-Pack or the LF03695, Basic Buddy® 10-Pack, (**refer to Figure 9**) for the proper packing arrangement of five fully assembled manikins into the nylon carrying bag (**LF03697**).



**Nasco Healthcare**  
16 Simulaids Drive  
Saugerties, NY 12477  
1-833-NASCOHC (627-2642)  
info@nascohealthcare.com  
www.nascohealthcare.com