

INSTRUCTION MANUAL

Basic Child CRiSis™ Manikin

LF03765



5 Year Warranty

Nasco
HEALTHCARE

About the Simulator



Figure 1

Basic Child CRiSis™ Manikin

This full-body child manikin has been created to offer realistic training and practice of child airway management and CPR education. The Manikin is anatomically correct in detail and represents the size of an 8-year-old child.

Realistic anatomy and landmarks include a mouth, tongue, oral pharynx, larynx, epiglottis, arytenoids, vocal cords, trachea, and esophagus. In addition to the palpable and visual landmarks, the head, neck, and jaw are fully articulated, allowing a realistic jaw thrust feature.

This manikin has separate left and right lungs for auscultation; has oral, nasal, and digital intubation capabilities; and includes an inflatable stomach bladder to indicate esophageal intubation. Also features E.T., E.O.A., P.T.L., L.M.A., E.G.T.A., and Combitube® insertion.

This basic airway management trainer may be gradually updated to the Life/form® Child CRiSis™ or Deluxe Child CRiSis™ manikins by adding the available upgrades

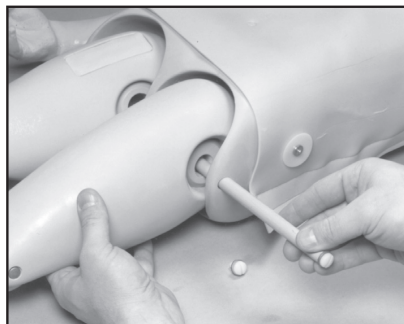


Figure 2

and the Interactive ECG Simulator (LF03670U). Includes pump spray lubricant. See page 4 for complete list of modular update components. Three-year warranty.

Assembly of Manikin

To facilitate packing, your manikin comes with the arms and legs unattached.

To connect the arms to the torso, place each in a position such that the hand is above the head. (**See figure 1.)**

Align the arm fitting with the torso socket and plug the arm into place. Finish by rotating the arm forward so it lies along the side of the body.

To connect the legs to the torso, simply slide the waist pin (supplied)

About the Simulator



through the torso and legs, pinning the legs in their correct position. (See figure 2.)

Features

Landmarks include nostrils, teeth, tongue, oral and nasal pharynx, larynx, epiglottis, arytenoids, false and true vocal cords, cricoid ring, tracheal rings, trachea, esophagus, lungs, and stomach.

Learners can practice oral, digital, and nasal intubation, as well as CPR. Suction techniques can also be performed and evaluated. The simulator was designed to use an uncuffed endotracheal tube measuring up to 5.0 mm I.D.

List of Components

- Child Airway Management Trainer Head
- Right and Left Lung Simulators
- Stomach Simulator
- Pump Spray Lubricant
- 3 Nylon Hose Clamps

Lubrication

Lubricate both the oral cavity and the endotracheal tube with the pump spray lubricant provided.

(See figure 3.) **Note:** Nasco Healthcare recommends the use of the provided lubricant or a similar vegetable-based lubricant for the Child Trainer. The use of a silicone-



Figure 3

based or similar lubricant may seriously damage the simulator, thus voiding warranty on the trainer.

Cleaning the Face

Normal soil can be removed from the face with mild soapy water. Nasco Cleaner (LF09919U) will remove stubborn stains. Simply apply Nasco Cleaner to soiled area and wipe clean with a soft cloth or paper towel. **Note:** Avoid using cleaner around mouth area if students will be applying direct mouth-to-mouth resuscitation techniques, as the cleaner may be toxic if ingested. NEVER place the trainer on any kind of printed paper or plastic. These materials, as well as ball-point pens, will transfer indelible stains. Do not use any cosmetics.

Available Supplies

- | | |
|-----------------|----------------------------------|
| LF03631U | Replacement Headskin with Airway |
| LF03627U | Replacement Lungs |
| LF03628U | Replacement Stomach |
| LF03644U | Pump Spray Lubricant |
| LF09919U | Nasco Cleaner |

Update Your Basic Life/form® Child CRiSis™ Manikin with Life/form® Modular Components

Modular Components

LF03611	Defibrillation Chest Skin
LF03612	Injectable Training Arm
LF03613	Blood Pressure Arm
LF03614	Intraosseous Infusion/ Femoral Access Leg
LF03618	Intraosseous Replacement Bones
LF03626	Femoral Replacement Injection Pad
LF03619	Intraosseous Replacement Skin
LF03683	Intraosseous Infusion Needle
LF03628	Replacement Stomach
LF03627	Replacement Lung Set Only
LF03670	Interactive ECG Simulator

Other Available Simulators

LF00698	Adult Injectable Arm (Light)	LF03000U	CPARLENE® Series
LF00855	Male Catheterization	LF03601U	Adult Airway Management Trainer with Stand
LF00856	Female Catheterization	LF03602U	Adult Airway Management Manikin
LF00901	Prostate Examination	LF03609U	Child Airway Management Trainer with Stand
LF00906	Ostomy Care	LF03616U	Child CRiSis™ Manikin
LF00929	Surgical Bandaging	LF03617U	Deluxe Child CRiSis™ Manikin with Arrhythmia Tutor
LF00957	Enema Administration	LF03620U	PALS Update Kit
LF00958	Pediatric Injectable Arm	LF03623U	Infant Airway Management Trainer with Stand
LF00961	Intramuscular Injection	LF03632U	Child Intraosseous Infusion/Femoral Access Leg on a Stand
LF00984	Breast Examination	LF03633U	Child Airway Management Trainer Torso
LF00995	Arterial Puncture Arm	LF03693U	Basic Buddy® CPR Manikin
LF00999	Pediatric Injectable Head	LF03699U	"Airway Larry" Airway Management Trainer
LF01005	First Aid Arm	LF03709U	Infant CRiSis™ Manikin
LF01008	Intradermal Injection Arm	LF03720U	Baby Buddy™ Infant CPR Manikin
LF01012	Heart Catheterization (TPN)	LF03750U	Bariatric CPR Manikin
LF01019	Ear Examination	LF03770U	Chest Tube
LF01027	Peritoneal Dialysis	LF03953U	CRiSis™ Manikin, Complete
LF01028	Suture Practice Arm	LF03955U	Deluxe CRiSis™ Manikin
LF01034	Suture Practice Leg	LF03956U	Deluxe "Plus" CRiSis™ Manikin
LF01036	Spinal Injection	LF03965U	Adult CRiSis™ Auscultation Manikin
LF01037	Hemodialysis Practice Arm	LF03966U	Adult CRiSis™ Auscultation Manikin with ECG Simulator
LF01038	Episiotomy Suturing Set	LF04000U	GERi™/KERi™ Manikin Series
LF01042	Suture Kit	LF04200U	Adult Sternal Intraosseous Infusion
LF01062	Pelvic, Normal & Abnormal	LF06001U	CPR Prompt® Adult/Child Manikin
LF01063	Stump Bandaging, Upper	LF06012U	CPR Prompt® Infant Manikin
LF01064	Stump Bandaging, Lower	LF06200U	CPR Prompt® Keychain Rescue Aid
LF01069	Cervical Effacement	LF06204U	CPR Prompt® Rescue and Practice Aid
LF01070	Birthing Station		
LF01082	Cricothyrotomy		
LF01083	Tracheostomy Care		
LF01084	Sigmoidoscopic Examination		
LF01087	Central Venous Cannulation		
LF01095	Blood Pressure Arm		
LF01108	Infant Intraosseous Infusion		
LF01121	Advanced IV Arm		
LF01131	Venipuncture and Injection Arm		
LF01139	Advanced IV Hand		
LF01142	Auscultation Trainer		
LF01143	Testicular Exam		
LF01152	Male & Female Catheter		
LF01155	Advanced CPR Dog		
LF01162	Venatech IV Trainer		
LF01174	NG Tube & Trach Skills		
LF01184	Venatech IM & Sub Q		
LF01193	Special Needs Baby		



Nasco Healthcare
16 Simulais Drive
Saugerties, NY 12477
1-833-NASCOHC (627-2642)
info@nascohealthcare.com
www.nascohealthcare.com