

Child Blood Pressure Arm Simulator LF03613U Instruction Manual

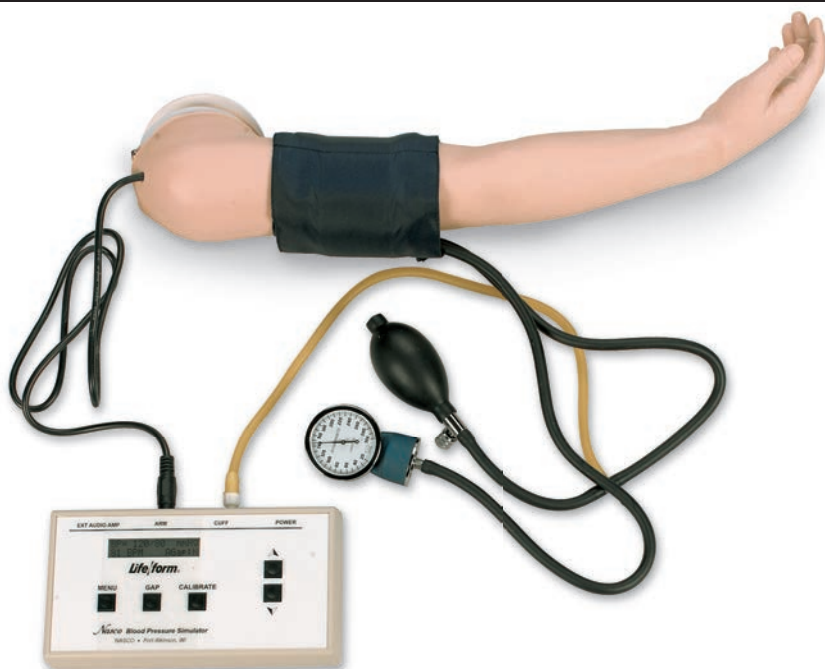

About the Simulator


The **Life/form®** Child Blood Pressure Arm is designed for years of maintenance-free operation as a training tool for not only the nurse, doctor, or medical technician, but also for anyone involved in the training of health care professionals.

The **Life/form®** Child Blood Pressure Arm has digitally recorded blood pressure sounds that can be varied by pulse rate and volume. The different Korotkoff phases can be identified, and an optional auscultatory gap can be included. A palpable radial pulse is present in the wrist. For additional uses, purchase the Amplifier/Speaker System (LF01189U) for group demonstrations or review.

List of Components

- Case
- Arm with Speaker Wire
- Electronic Control Unit
- Six "AA" Batteries
- Sphygmomanometer

Attaching the Arm to Your Resusci® Junior* Manikin

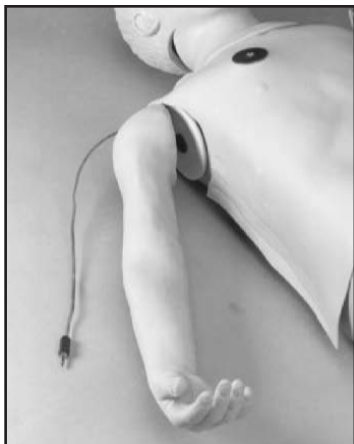

1. Remove the existing arm by loosening the six screws on the body on both sides of the right arm.



Figure 1

2. Spread the body halves, then pull the arm and retaining ring out. (**See figure 1.**)
3. Pull the retaining ring apart and remove it from the arm.



Figure 2

4. Reassemble the retaining ring through the hole in the shoulder assembly of the Child Blood Pressure Arm. (**See figure 2.**)
5. Reinsert the arm in the body and replace the six screws.

NOTE: If this arm is a replacement for the Nasco Child **CRiSis™** Manikin, simply remove the old arm by rotating it above the head and pulling out at the shoulder. Install the new arm by reversing this procedure.

***RESUSCI® JUNIOR IS A TRADEMARK OF THE LAERDAL CORPORATION.**

General Instructions for Use Installing the Batteries

Take the Blood Pressure Electronic Control Unit from the box and turn it over, placing it face down onto a padded work surface. Locate the "Open" compartment on the back of the panel where the batteries are to be installed. **(See figure 3.)** Place your thumb or index finger on the "Open" compartment and push up.

This will open the battery compartment. The compartment is marked as to the "+" and "-" positions of the batteries. The battery bracket is now accessible to the user.



Figure 3

Install six "AA" batteries as indicated by the orientation diagram embossed in the bottom of the bracket. It is recommended that alkaline batteries be used for increased battery life. After the batteries have been properly installed, reassemble the Blood Pressure Simulator by simply reversing the disassembly procedures. Place the unit face up on the work surface and turn it on by pressing the on/off switch on the top right of the unit. **(See figure 4.)** Observe the display and verify that a readable display is present, indicating proper battery installation.

NOTE: The control box has a battery saving feature which will turn the unit off after about 8-10 minutes if no keys are used within that period of time.

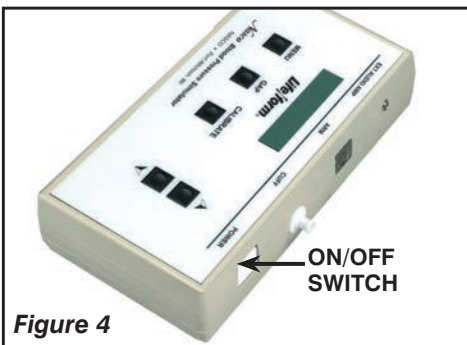


Figure 4

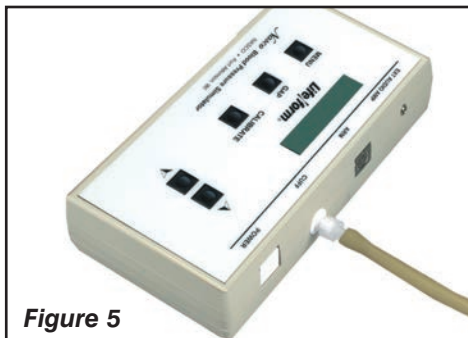


Figure 5

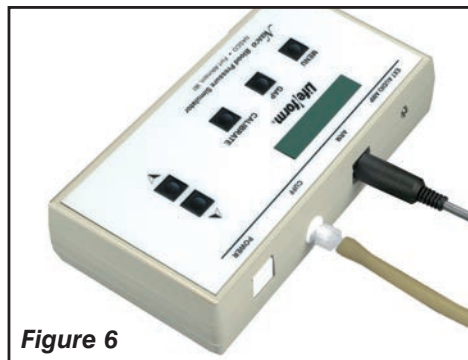


Figure 6

The next step is to connect the simulated arm and speaker assembly along with the sphygmomanometer cuff and gauge assembly included with the unit. First, locate the end of the pressure line attached to the sphygmomanometer that has the male luer fitting attached to it. Attach this to the female luer fitting at the top of the unit marked CUFF. **(See figure 5.)** After the pressure line fitting has been properly installed, locate the plug that is at the end of the wire which extends from the simulated arm assembly. Insert the plug from the arm into the jack at the top of the unit marked ARM. **(See figure 6.)** At this point the Blood Pressure Simulator is ready for use. The unit has been factory calibrated for use with the accessories included in the kit. No further calibration adjustments should be necessary at this time. If the unit is to be used with a sphygmomanometer other than that supplied with the unit, or if recalibration is necessary at a later date, then see the section titled "Calibration Procedures".

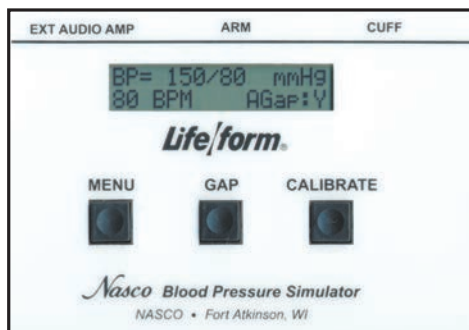


Figure 7

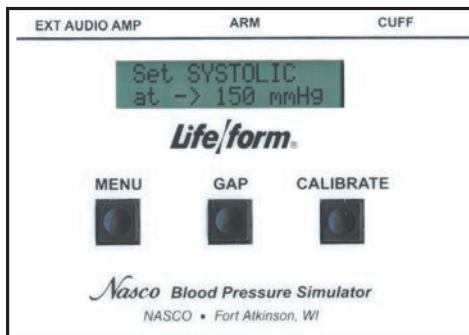


Figure 8

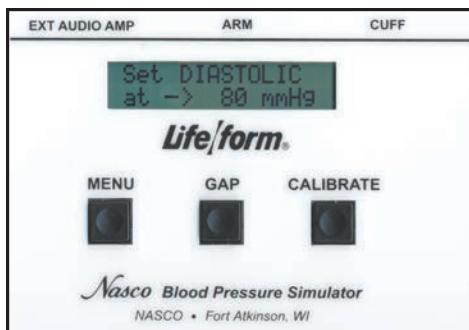


Figure 9

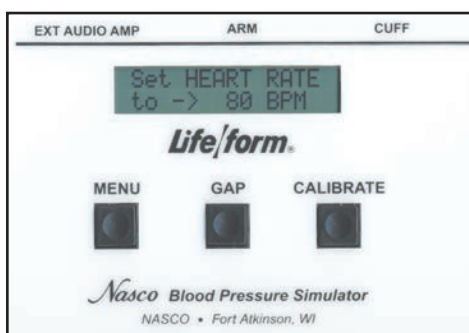


Figure 10

Familiarizing Yourself with the Nasco **Life/form**® Child Blood Pressure Simulator Control Panel

Under the display window are three buttons: Menu, Gap, and Calibrate. (See figure 7.) The systolic pressure is set by pressing the Menu key once. The pressure is adjusted up or down using the up and down arrow keys. (See figure 8.) The diastolic pressure is set by pressing the Menu key a second time. Adjust the setting up or down with the arrow up and down keys. (See figure 9.) The heart rate is set by pressing the Menu key a third time, and adjusting the rate with the arrow up and down keys. The pulse rate can be set from 0 beats per minute to 300 beats per minute. (See figure 10.) The palpation can be set to either on or pulseless. When the pulseless setting is used, the diastolic and systolic pressures will automatically be set to 0.

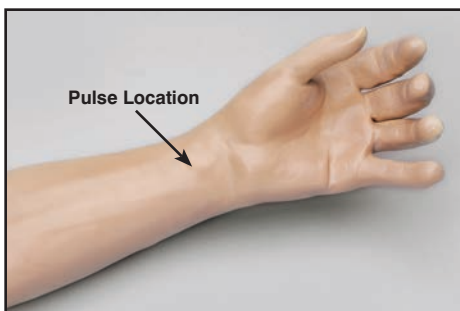


Figure 11

Palpable Pulse Feature

The Nasco **Life/form**® Child Blood Pressure Simulator also incorporates a palpable pulse at the radial location. (See figure 11.) Palpations can be felt upon start-up of the unit or after blood pressure settings have been made. Press the Menu key repeatedly until "Set PALPATION" menu appears. "Pulse ON" is defaulted and enables the palpation feature. The pulse in the wrist will begin when the electronic box is turned on. During the actual blood pressure reading, the palpable pulse will turn off when the cuff is inflated and surpasses the systolic set point. It will turn on again when the cuff is deflated 20 mmHg BELOW the diastolic set point.

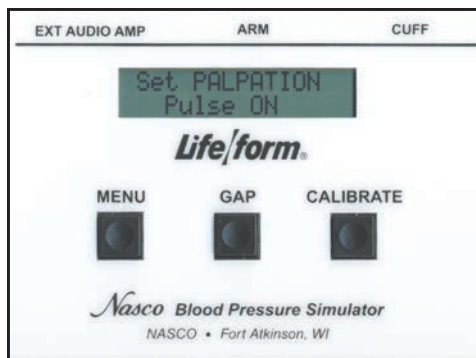


Figure 12

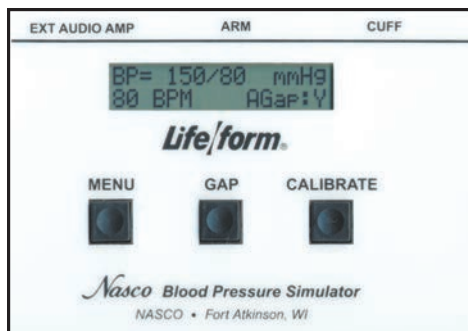


Figure 13

NOTE: The pulse feature is turned off during this time to allow students to clearly hear the Korotkoff sounds through the stethoscope without interference from the background noise of the pulse feature in the wrist. **The palpable pulse is delicate and should be palpated lightly as you would with a real patient. Pressing too hard can damage the pulse feature.**

Make sure the cable from the blood pressure arm is properly connected to the electronic box and the Palpation feature is in the "PULSE ON" mode.

A pulseless condition can be simulated by switching your unit to the "pulseless" mode. Press the Menu key repeatedly until "Set PALPATION" menu appears. By pressing the down arrow at this point, palpations can be disabled, causing the simulator to be pulseless. When in the "pulseless" mode, all settings are automatically reset to 0 and all blood pressure sounds are disabled.

The pulseless setting will also turn off the sounds in the arm. The pulse will always be on unless the pulseless feature is activated or if the systolic or heart rate levels are set to zero. To do this, press the Menu key four times. The down arrow key will set the pulse to pulseless. Press the arrow up key to turn the pulse back on. (**See figure 12.**) Located to the right of the Menu key is the auscultatory Gap key. (**See figure 13.**) This key is included to simulate the auscultatory gap that is sometimes present between phase 1 and phase 2 sounds in which no audible sound is noted during this portion of auscultation. This control function is included so that the trainee can become familiar with this phenomenon. Pressing the Gap key simply turns the gap function off or on. When the key is pressed, a message will briefly appear that the auscultatory gap is enabled or disabled. Also the main display will show (at the bottom right of the display) either AGap:Y (for on) or AGap:N (for off).

The arrow up and down keys also control the volume of the sounds that are present in the arm. From the main menu, press the up arrow key to increase the volume, press the down arrow key to decrease the volume. The volume levels can be adjusted from level 1 (the lowest volume) to level 7 (the highest volume).

Using the Nasco *Life/form*® Child Blood Pressure Simulator

First, verify that the pressure line tubing from the sphygmomanometer and the audio line coming from the simulated arm assembly are properly connected to the blood pressure simulator unit, as previously described in the set up procedures. Apply the sphygmomanometer cuff and gauge to the simulated arm assembly in the usual manner. Apply the stethoscope to the simulated arm, also in the usual manner. Set the systolic and diastolic controls to the desired levels. Select the auscultatory gap if desired. Finally, adjust the pulse rate control to the desired setting.

To proceed with the simulated blood pressure measurement, first close the valve on the sphygmomanometer bulb tightly and begin pumping air into the cuff until the gauge reads higher than the preset systolic level chosen. Once this point is reached, loosen the valve on the bulb slightly to allow the gauge pressure reading to decrease slowly. While monitoring the arm assembly with the stethoscope, note the point on the sphygmomanometer gauge when the first Korotkoff sound is heard. This will be the systolic blood pressure. Allow the pressure in the cuff to continue to decrease until the point at which the last pulse is heard, noting the reading on the gauge. This is the diastolic blood pressure.

Compare the results of reading the systolic and diastolic blood pressures on the gauge with the respective settings on the simulator. If the readings were accurate, they should compare favorably with the preset values. If the auscultatory gap had been selected, then an absence of an audible pulse would have been noticed during what would have been the phase 2 Korotkoff sound. It is this lack of an audible pulse that is considered an auscultatory gap. Note that in reality the auscultatory gap can be present in either the phase 1 or phase 2 Korotkoff sounds. If it is desired to demonstrate the sounds heard while measuring a subject's blood pressure to the trainee or group of trainees, an auxiliary amplifier/speaker system is available (LF01189U). If the auxiliary speaker is used, the speaker is plugged into the EXT AUDIO AMP jack located next to the ARM output jack at the top of the unit. Adjust the volume control to increase the output of the auxiliary amplifier/speaker, and proceed with the blood pressure measurement sequence as it would normally be performed, except that the stethoscope need not be used. Instead, listen to the sounds as they emanate from the amplifier/speaker, noting the differences in the Korotkoff phases being presented.

The Nasco **Life/form**® Child Blood Pressure Simulator is programmed to

demonstrate the 5 Korotkoff phases, including an auscultatory gap, which can be heard during auscultation of a subject while measuring the subject's blood pressure. Each is distinctly different and present for only a portion of the measurement sequence.

Low Battery Indicator

When the battery supply diminishes to a level near the point that the unit will no longer function properly, the "low batt" segment of the systolic pressure display will activate when the pressure in the sphygmomanometer cuff reaches above 20 mmHg. At this point, the batteries should be replaced as soon as possible in order to insure proper operation of the unit. Refer to the section titled "Installing the Batteries."

Calibration Procedures

To calibrate the simulator, set the unit up as described in the section titled "Using The Nasco **Life/form**® Child Blood Pressure Simulator." Apply the cuff to the simulated arm. Set the control box systolic pressure to 150 mmHg and set the diastolic pressure to 70 mmHg. Proceed with a simulated blood pressure measurement. Note the discrepancy in the readings between the gauge and the control box. Set the systolic "offset."

Example: If the blood pressure reading was taken and the sounds started at 148 mmHg, then the offset is +2. If the sounds started at 152 mmHg, the offset is -2. For this example, assume that the sounds started at 148 mmHg. Press and hold the Calibration key until the systolic correction window appears. (**See figure 14.**) Using the arrow up key set the correction to +2. Set the diastolic offset.

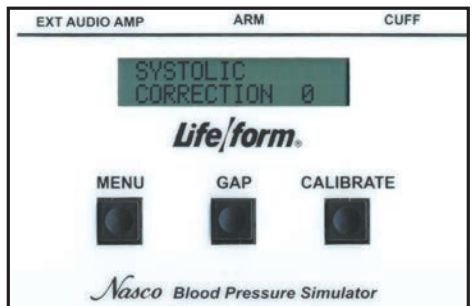


Figure 14

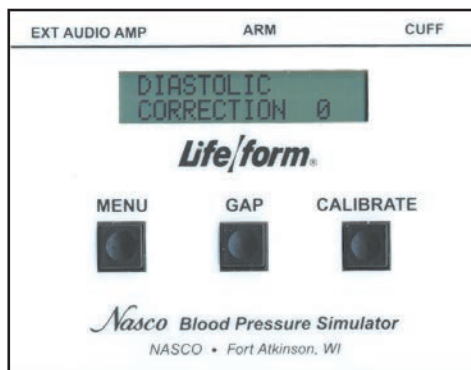


Figure 15

Example: If the blood pressure reading was taken and the sounds stopped at 72 mmHg, the offset is -2; or if the sounds stopped at 68 mmHg, the offset is +2. For this example, assume that the sounds stopped at 72 mmHg. From the systolic window, press the Menu key to change the diastolic window. **(See figure 15.)** With the arrow down key, set the correction to -2. Press the Menu key again and the message “CALIBRATION COMPLETE” will appear. The main menu window will be displayed.

Preparing Your Equipment for Use with the Nasco *Life/form*® Child Blood Pressure Simulator

To adapt your sphygmomanometer for use with the simulator, it is first necessary to obtain the luer fitting and the T-fitting included with the simulator, which is also available through your Nasco catalog sales office. Using a wire cutting pliers or similar instrument, carefully cut the pressure line of the sphygmomanometer about 2" from the gauge. **(See figure 16.)**

Take the T-fitting and insert it between the two ends of tubing that were previously cut. **(See figure 17.)** Assemble the portion of pressure line tubing over the remaining barbed end on the T-fitting. **(See figure 18.)**



Figure 16



Figure 17



Figure 18

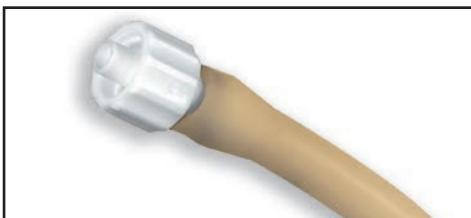


Figure 19

Lastly, take the loose end of the tubing and push the male luer fitting into it. **(See figure 19.)** Attach the male luer fitting to the fitting marked CUFF at the top of the unit. Once all of the necessary connections have been properly made, go to the section titled “Calibration Procedures” and calibrate the unit. When the calibration procedures have been completed, the simulator is ready for use.

Supplies/Replacement Parts for Child Blood Pressure Simulator

LF01096U Electronic Control Unit with Sphygmomanometer

LF01189U Amplifier/Speaker System

Other Available *Life/form*® Simulators

LF00698U	Adult Injectable Arm (White)	LF01184U	Venatech IM & Sub Q
LF00855U	Male Catheterization	LF01193U	Special Needs Baby
LF00856U	Female Catheterization	LF03000U	CPARLENE ® Series
LF00901U	Prostate Examination	LF03601U	Adult Airway Management Trainer with Stand
LF00906U	Ostomy Care	LF03602U	Adult Airway Management Manikin
LF00929U	Surgical Bandaging	LF03609U	Child Airway Management Trainer with Stand
LF00957U	Enema Administration	LF03616U	Child CRiSis ™ Manikin
LF00958U	Pediatric Injectable Arm	LF03617U	Deluxe Child CRiSis ™ Manikin with Arrhythmia Tutor
LF00961U	Intramuscular Injection	LF03620U	PALS Update Kit
LF00984U	Breast Examination	LF03623U	Infant Airway Management Trainer with Stand
LF00995U	Arterial Puncture Arm	LF03632U	Child Intraosseous Infusion/ Femoral Access Leg on a Stand
LF00999U	Pediatric Injectable Head	LF03633U	Child Airway Management Trainer Torso
LF01005U	First Aid Arm	LF03693U	Basic Buddy ® CPR Manikin
LF01008U	Intradermal Injection Arm	LF03699U	"Airway Larry" Airway Management Trainer
LF01012U	Heart Catheterization (TPN)	LF03709U	Infant CRiSis ™ Manikin
LF01019U	Ear Examination	LF03720U	Baby Buddy ™ Infant CPR Manikin
LF01027U	Peritoneal Dialysis	LF03750U	Fat Old Fred
LF01028U	Suture Practice Arm	LF03760U	Airway Management/Cricoid Pressure Trainer
LF01034U	Suture Practice Leg	LF03770U	Chest Tube
LF01036U	Spinal Injection	LF03953U	CRiSis ™ Manikin, Complete
LF01037U	Hemodialysis Practice Arm	LF03955U	Deluxe CRiSis ™ Manikin
LF01038U	Episiotomy Suturing Set	LF03956U	Deluxe "Plus" CRiSis ™ Manikin
LF01042U	Suture Kit	LF03965U	Adult CRiSis ™ Auscultation Manikin
LF01062U	Pelvic, Normal & Abnormal	LF03966U	Adult CRiSis ™ Auscultation Manikin with ECG Simulator
LF01063U	Stump Bandaging, Upper	LF04000U	GERI ™ / KERI ™ Manikin Series
LF01064U	Stump Bandaging, Lower	LF04200U	Adult Sternal Intraosseous Infusion
LF01069U	Cervical Effacement	LF06001U	CPR Prompt® Adult/Child Manikin
LF01070U	Birth Station	LF06012U	CPR Prompt® Infant Manikin
LF01082U	Cricothyrotomy	LF06200U	CPR Prompt® Keychain Rescue Aid
LF01083U	Tracheostomy Care	LF06204U	CPR Prompt® Rescue and Practice Aid
LF01084U	Sigmoidoscopic Examination		
LF01087U	Central Venous Cannulation		
LF01095U	Blood Pressure Arm		
LF01108U	Infant Intraosseous Infusion		
LF01121U	Advanced IV Arm		
LF01131U	Venipuncture and Injection Arm		
LF01139U	Advanced IV Hand		
LF01142U	Auscultation Trainer		
LF01143U	Testicular Exam		
LF01152U	Male & Female Catheter		
LF01155U	Advanced CPR Dog		
LF01162U	Venatech IV Trainer		
LF01174U	NG Tube & Trach Skills		

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1-800-558-9595

eNasco.com • E-mail: lifeform@eNasco.com