

USER GUIDE

Advanced BLS Simulator

(BT-CPEA-PLUS)




1. Cautions

- **Please be sure to read this user guide carefully since it is for appropriate use of product and safety of users. Manufacturer is not liable for any accidents caused by failure to observe this instruction.**
- ① Use components only provided or approved by BT.
- ② Do not use product for any other purpose described in this manual.
- ③ In case of any disassembly or modification of product without permission by BT, we are not going to be offered any customer service. Also, imitation or remodeling for commercial purpose is prohibited by law.
- ④ During the usage, please contact us after any electricity power off when the product is overheated or smelled something burning.
- ⑤ Do not install other programs or software to the provided laptop. It may cause malfunction of software.
- ⑥ Keep product away from water and moisture. If the product gets wet, do not turn it on since it may cause an electric shock.
- ⑦ Do not use the product near strong magnetic fields or high voltage power lines.
The product may malfunction due to the electromagnetic waves.
- ⑧ Do not use the product under a weather of lightning. It can cause the product malfunction and increases the risk of electric shock.
- ⑨ Do not use product with extreme temperature.
(Operating temperature : 0°C ~ 40°C,
Storage temperature : -10°C ~ 50°C)
- ⑩ Do not keep the product near hot or cold places. Deterioration of product or internal damage may be occurred.
- ⑪ Never clean the product with chemical substances such as solvent, benzene or thinner, as these may cause ignition or the surface deterioration.
- ⑫ Product specification may be changed without notice in order to improve product.


2. Contents

1. Cautions	1
2. Contents	2
3. Components	3
4. Features & Functions	4
5. Hardware	
5-1 Install electrode covers	5
5-2 Connect Arms & Legs & Pelvis	6
5-3 Functions of manikin	7
6. PC Setting, Installation & Update	
6-1 PC Setting, Check network and bluetooth	8
6-2 Install & update	10
7. Basic functions	
7-1 Screen Layout	11
7-2 Menu and other icons	12
8. Practice	
8-1 Add device	16
8-2 Set file path and Register trainee	17
8-3 Settings	17
8-4 Match trainee's information	18
8-5 Practice	18
8-6 Scenario	19
8-7 Patient monitor	19
8-8 Metronome	21
8-9 Checklist	21
8-10 Accuracy Analysis	22
8-11 Evaluation result	23
8-12 Previous result	24
9. Convenient functions	25

3. Components

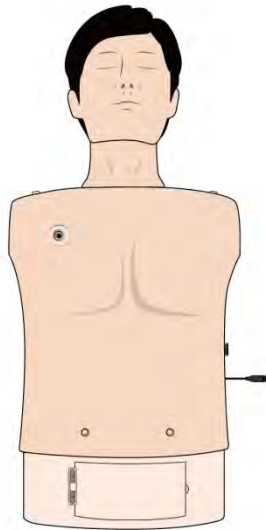


Manikin (Upper Body)

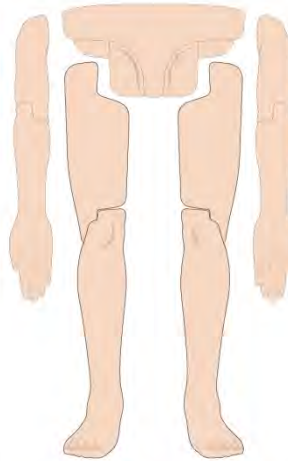

Arms / Legs / Pelvis (1 Pair Each)



Laptop Computer
(With software)

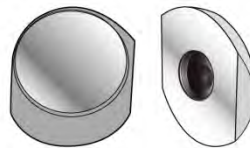

Defibrillator Electrode Cover (2EA)




Power converter




Suits (Jacket and Pants)

※ etc. – Storage case, User guide

4. Features & Functions

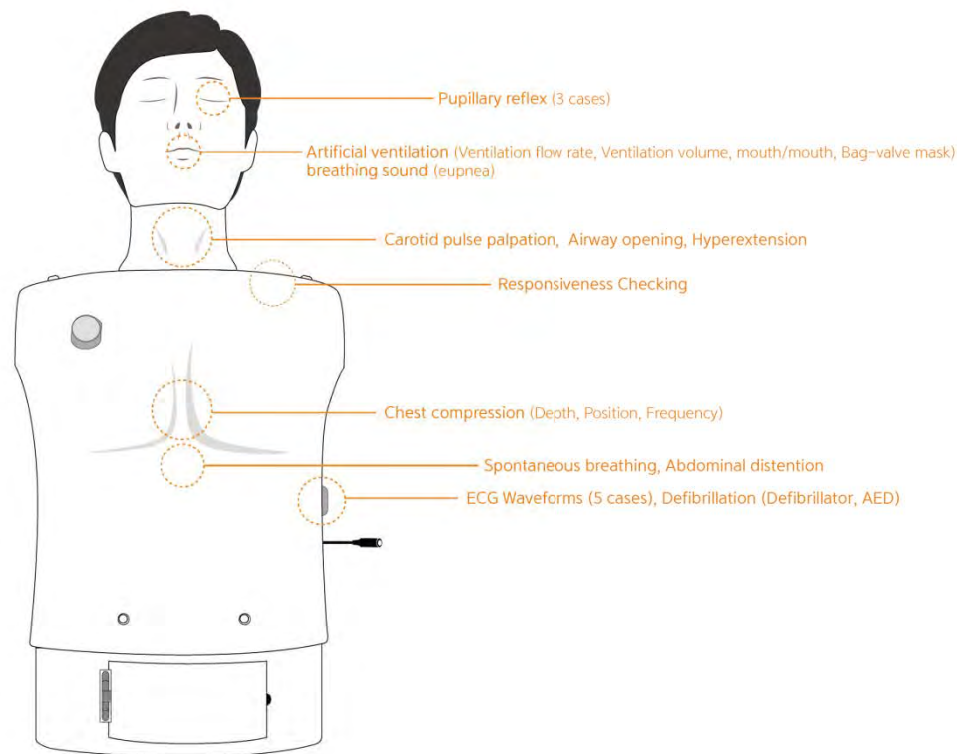


< Functions of manikin >

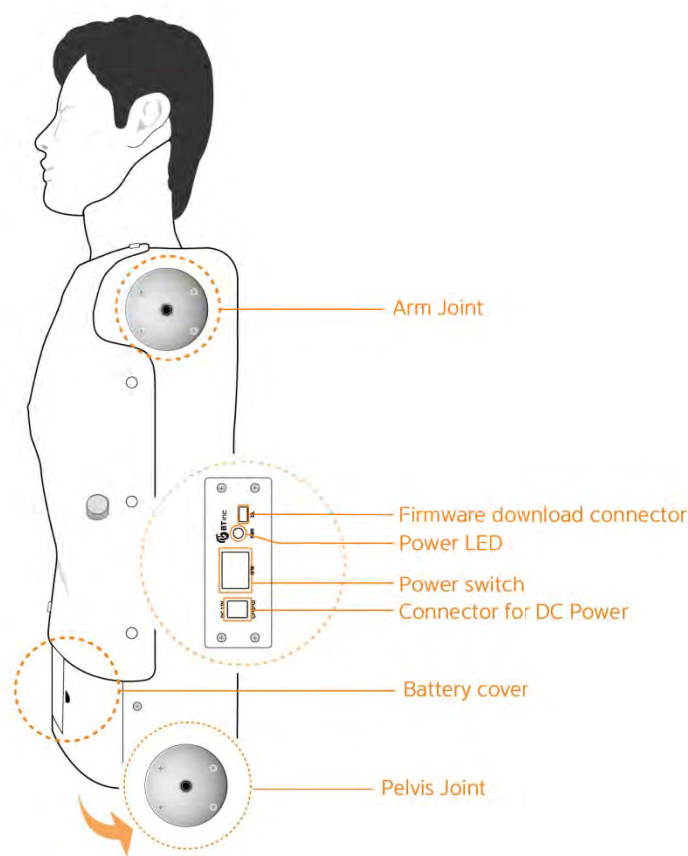


< Power supply and connections >

5. Hardware

5-1. Install electrode covers

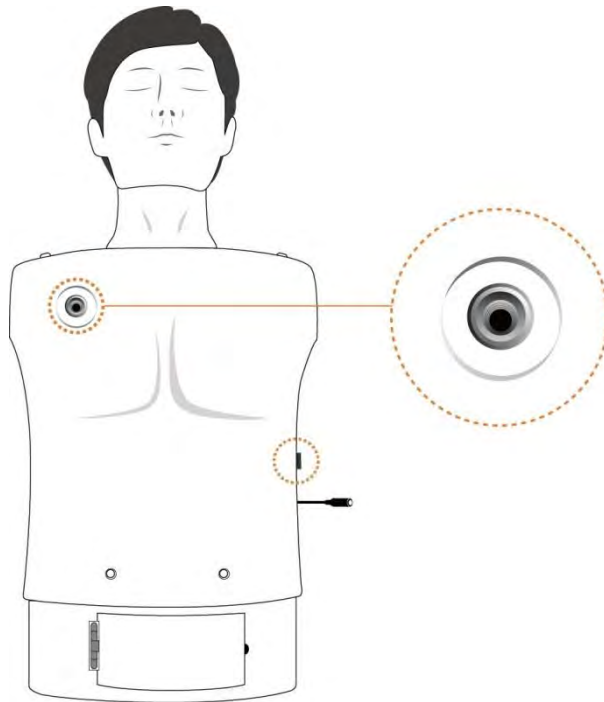


Install electrode covers to the connectors at RA(Right arm) & LL(Left leg) of manikin.

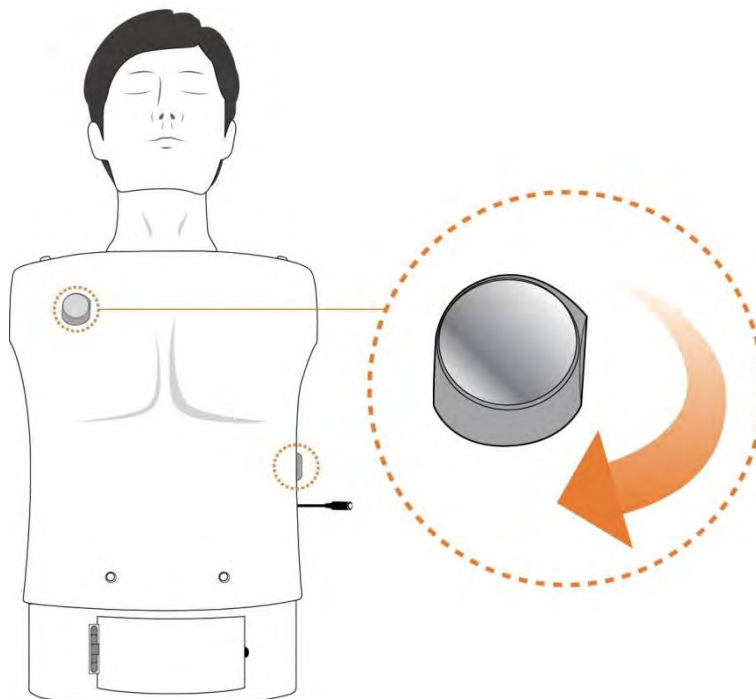

Attach the electrode cover on the threaded connectors and turn it clockwise to install.
Turn it counterclockwise to uninstall.

⚠ Caution : Be sure to install electrode covers before using AED pads.

5-2 Connect Arms & Legs & Pelvis

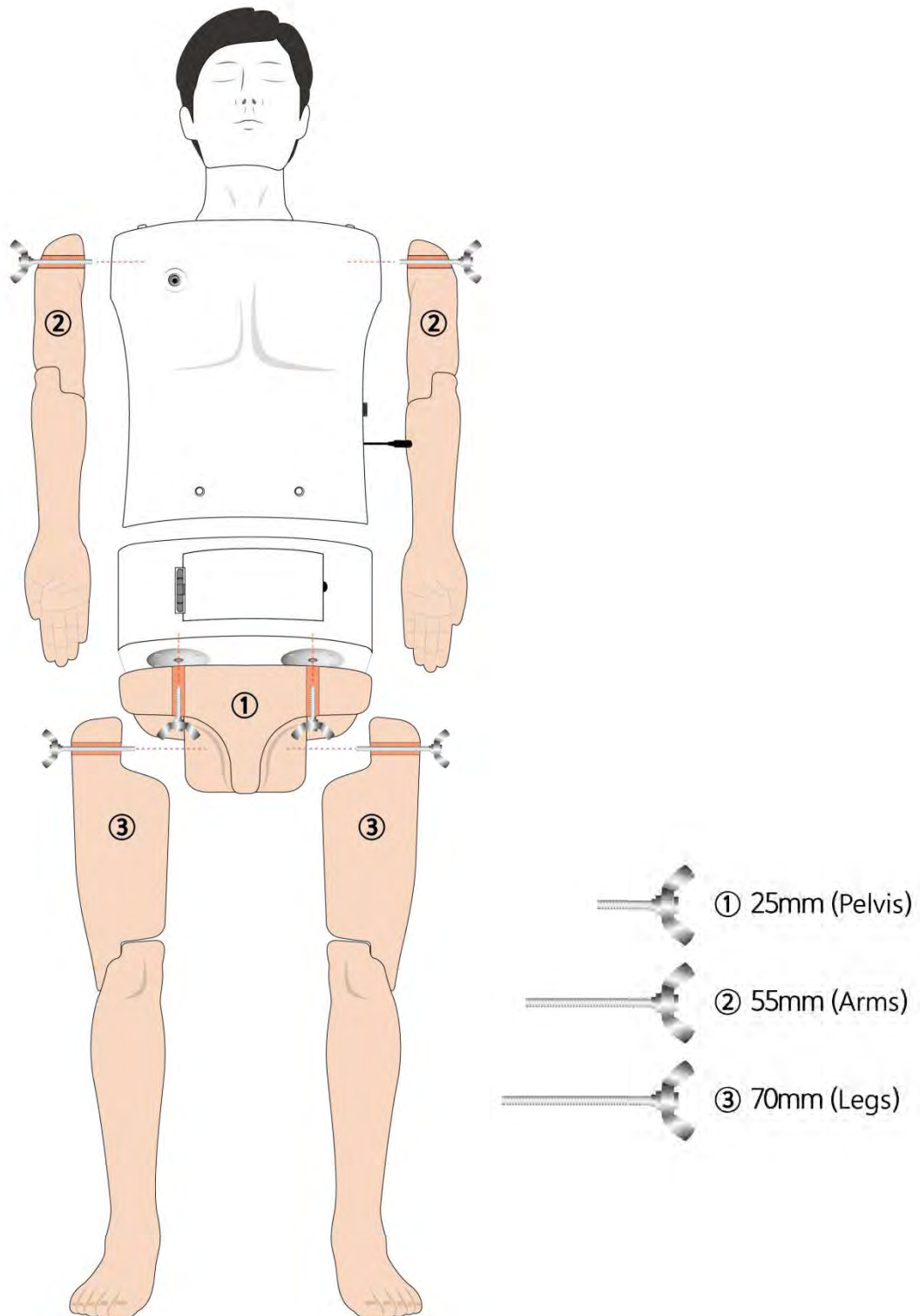


- Place each arms, legs, and pelvis parts at the joint and assemble these by screwing bolts with hands.


5-3 Functions of manikin



Checking responsiveness

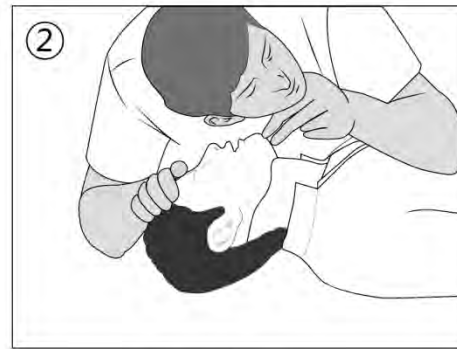


Checking Breathing

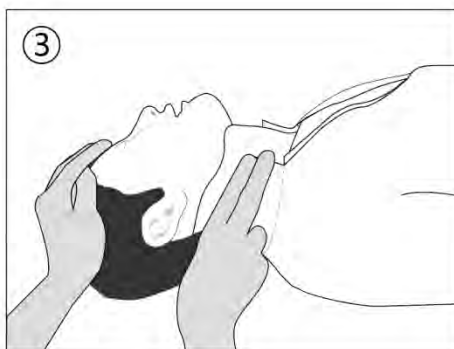


Checking Pulse



Checking Pupil

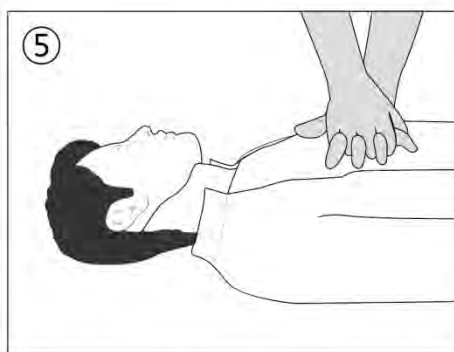

Chest compression

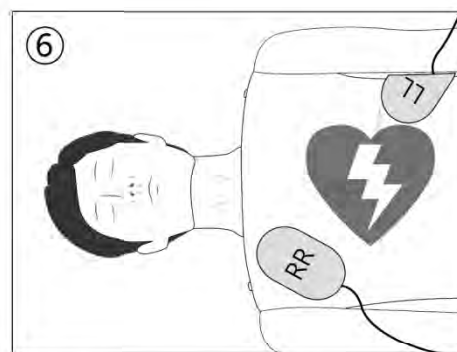

Defibrillation (Electric shock)



Opening Airway




Artificial ventilation

6. PC Setting, Installation & Update

6-1. PC Setting, Check network and bluetooth

Please turn-off the screen saver and sleep mode of computer.


- ① Click **[Action center]** at the bottom of main screen.

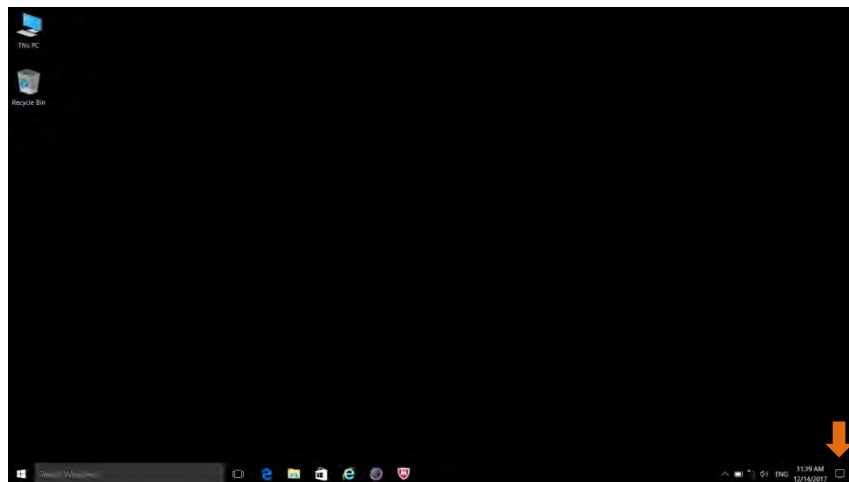


<Main screen>

Action center

- ② Click **[Expand]**.

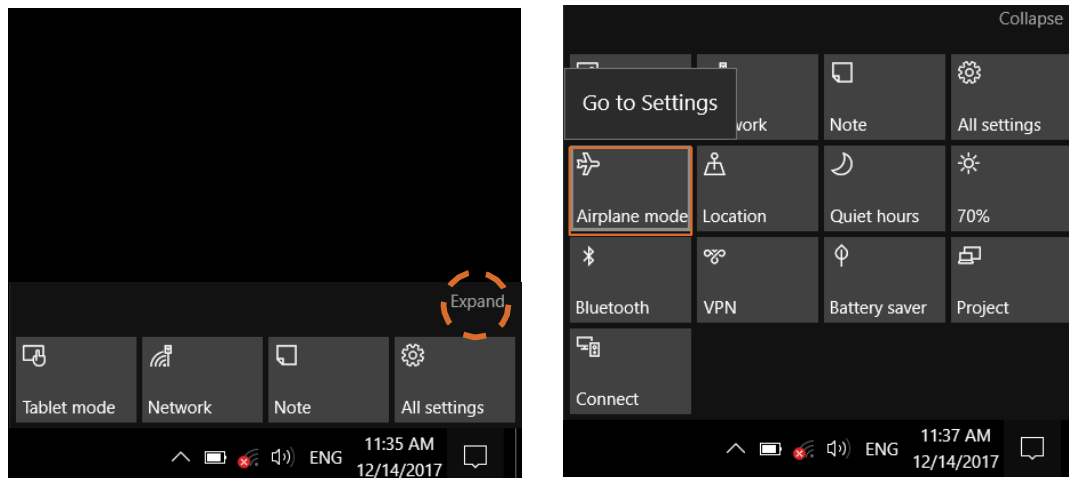

Action center>Expand>Airplane mode> 'Right click'>Go to Setting

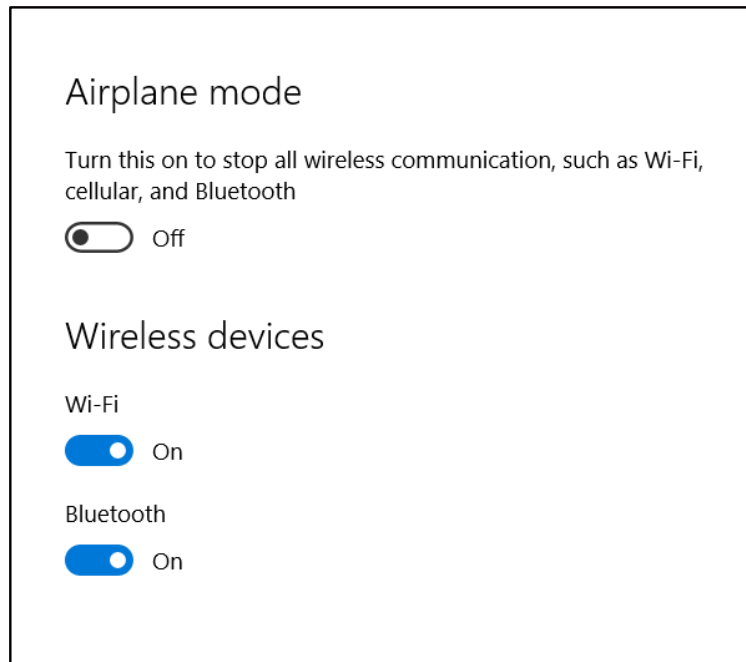

- ③ Click **[Airplane mode]** and see settings is **[Bluetooth ON]**, **[Wi-Fi ON]**, **[Airplane mode OFF]**.
If not, please set things refer to this.


- ④ After setting, go to main screen by using shortcut key, **[Windows icon  + D]**.

*** Windows8, 8.1**


[Windows icon  + C], go to Search → Network → Airplane mode.
Make sure settings is [Bluetooth ON], [Wi-Fi ON], [Airplane mode OFF].

6-2. Install & Update

Users can download S/W from BT webpage. (<http://www.btinc.co.kr>)

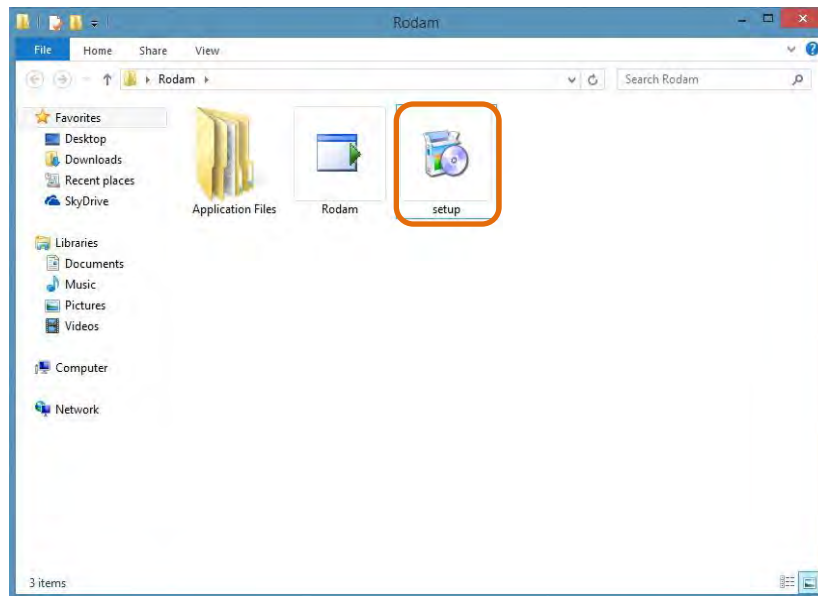

① Double-click 'setup' file and start installation.

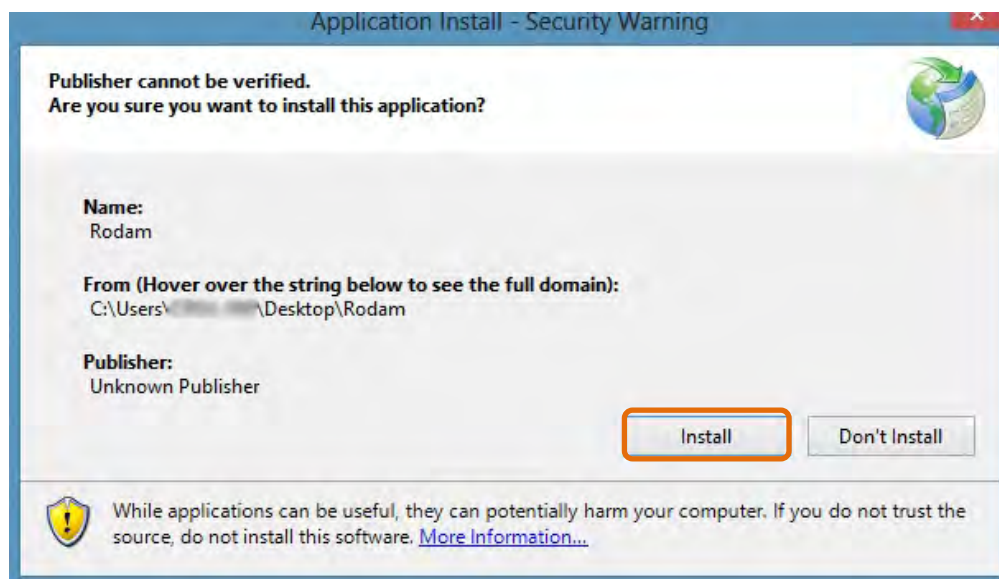

② Click 'Install' and proceed installation. S/W will be opened when installation is done.



- Installation is based on Windows 10 and setting procedure may be different from Windows version.


In case of Windows 7, 8, 8., internet connection might be needed to install NET Framework.

7. Basic functions

7-1. Screen layout

RoDam (BT-CPEA) program main screen will appear as below.

This figure explains the layout of initial screen. There may be difference of layout depended on user settings.

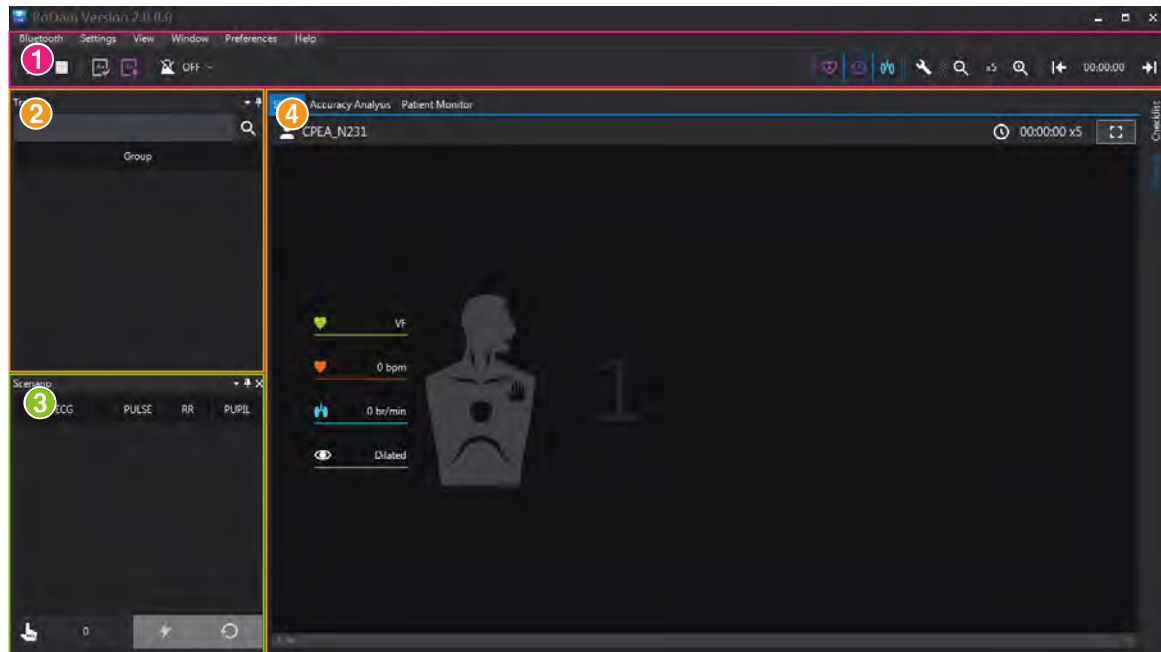

Users can customize the size of template.

- ❶ Menu : Bluetooth, Settings, View, Windows, Help menus are offered.
- ❷ Trainee : User can search the trainee's information at the list and drag&drop the trainee's information to the selected manikin in order to load and start training.


* To disconnect, double-click the trainee's information at the list of trainee.
- ❸ Scenarios : The status of manikin is changed with an action of defibrillation.
- ❹ (1) Training Screen : Multiple screen will be generated based on the number of connected manikins and it is possible to check chest compression, ventilation and history log. In case of multi training, 'view all menu' and 'view individual menu' is provided so that trainers can monitor easily.
(2)Accuracy analysis : User can check the detailed information about compression & respiration with graphic charts.
(3)Patient monitor : Users can check the vital sign of patient.
(4)History : Automated check list, Check list made by user can be effected to the score.
(5)Checklist : The checklist which is designed by users will be displayed. Checking and unchecking the list are possible during practice or after practice.

7-2. Menu and other icons

User can check the Icons on the top of the RoDam(BT-CPEA) program.

In this user manual, 'Bluetooth', 'Settings', 'View', 'Window', 'Preference', 'Help' will be introduced.

Menu – Practice icon

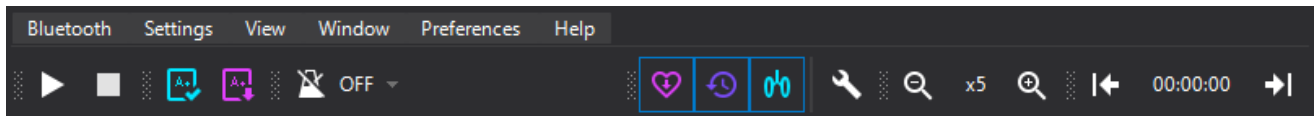

ICONS	
	Start training
	End Training
	See result
	See Previous result
	Metronome
	Show chest compression graph
	Show history
	Show ventilation graph
	Graph option
	Zoom out graph
	Zoom in graph
	Move graph to left
	Move graph to right

※ Additional icons

- ①  Previous result >  Statistics >  Export statistics into Excel format
 Item selector for display
- ②  Result >  Trainee information >  Set size of logo,  Original size of logo
 Open logo
- ③ Right upper side of screen >  Full screen
 Previous screen

Users can customize screens according to various training patterns.

Bluetooth Icon

Bluetooth	Setting
 Search	Search Search for available manikins
 Add	Add Add appropriate manikins
 Remove	Remove Disconnect manikins

Setting Icon

Settings	View	Window
 Registration		Register trainee/ Select a folder to save result
 Guideline		Guide line setting (Chest compression, ventilation, etc.)
 Scenario		Make scenario
 Assignment Scoring		Setting evaluation score (Chest compression, ventilation)
 Checklist		Checklist list and score setting
 Practice Time		Training time

⌘ Additional icon

①  Registration >  Folder to save result

Please set “Settings” before training.



- Maximum 7 manikins can be connected to 1 laptop by Bluetooth.
- The latest setting will be stored automatically for next training.

View Icon

View

Window

Preferen

 Placement

 History

 Patient Monitor



Manikin layout settings for multi-training



ON/OFF Automated/manual check list



Show Patient monitor

Users can customize screens according to various training patterns.

- 
- The layout of main screen can be customized before, during and after training.

- The latest setting will be stored automatically for next training.

- 'At 'Previous result' menu, screen layout setting of 'View' menu is partly available.

Window Icon

Window

Preferences

Help

 Reset Window Layout

 Trainee

 Scenario

 Checklist

 History

 Accuracy Analysis

 Patient Monitor



Reset window layout
(Reset to the initial setting)



Show 'Trainee' layout

Show 'Scenario' layout


Show 'Checklist' layout

Show 'History' layout


Show 'Accuracy Analysis' layout

Show 'Patient Monitor' layout

Preference Icon

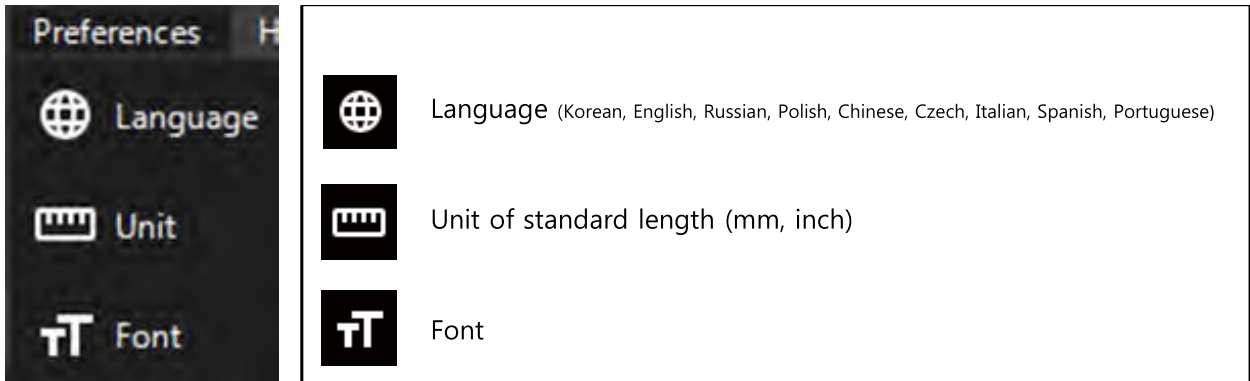

Help Icon

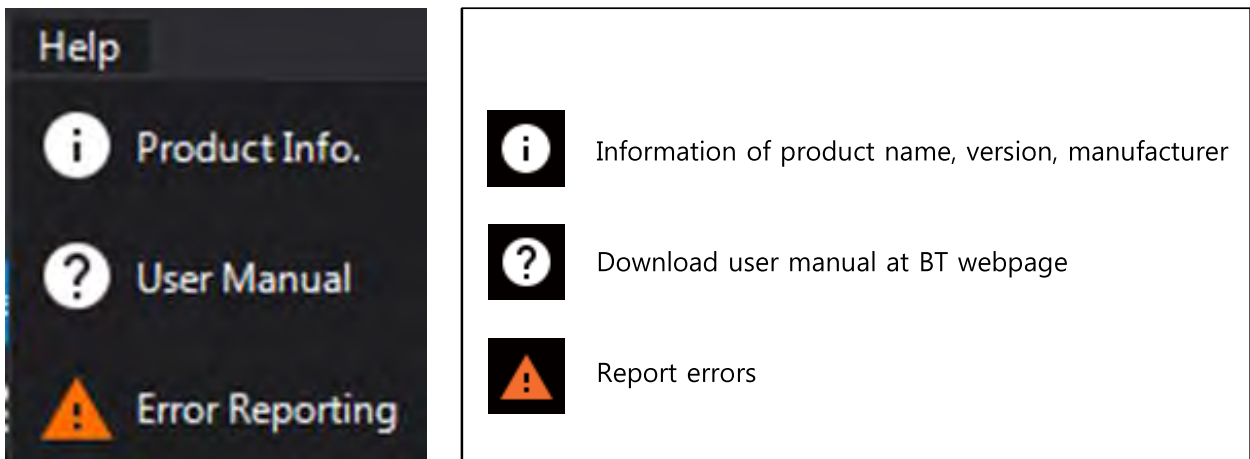


8. Practice


8-1 Add device

Available manikin will be appeared on the screen after searching for manikin.


 - Manikin has to be connected to power to do all procedures.

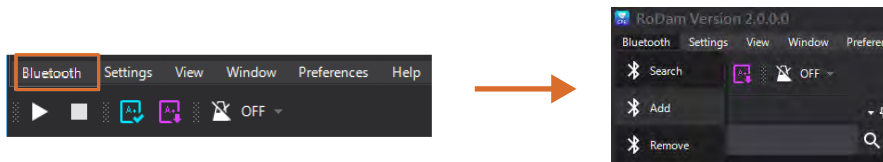

① Choose a device to add by selecting 'Bluetooth > Add' at the top of screen.

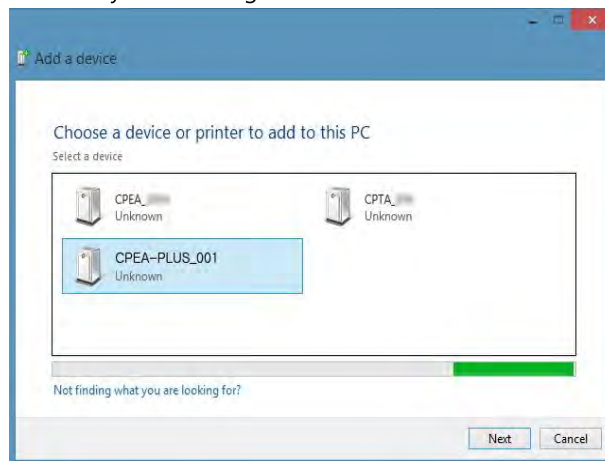

 - Please refer to a label at the side of manikin to check the name of device.

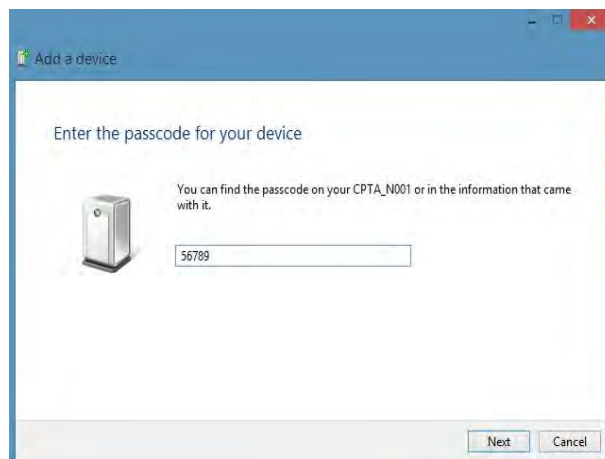


② When the device is appeared on the screen, pair manikin and computer by typing pin code.

 - Pin Code : 56789

③ To apply modification, please restart program by selecting 'Bluetooth > Add' menu. Users can find the device id connected after restarting S/W.


8-2. Set file path and Register trainee

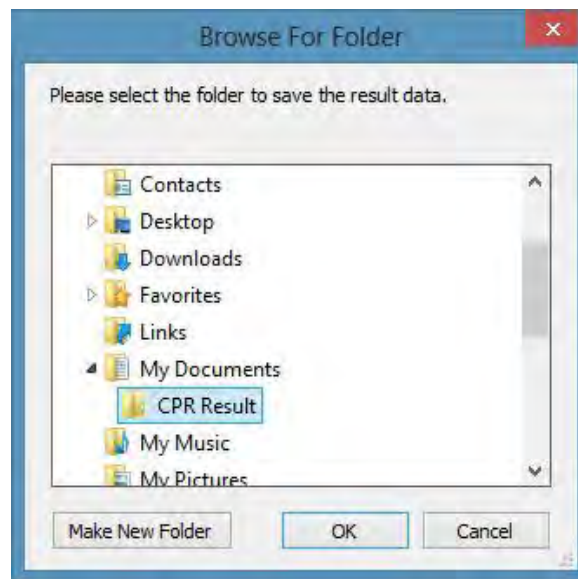

8-2-1. Set file path for result files



- Select a folder that result files will be saved at 'Settings>Registration>File path' menu.
- The name of folder will be automatically set as 'CPR Result'.

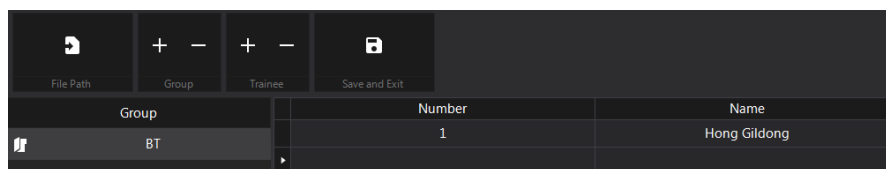


8-2-2. Register group and trainee



- After creating a 'Group', save 'Trainee information' after typing it.
- 'Remark' is not a mandatory but it will displayed at the result screen if it is typed.

8-3. Settings

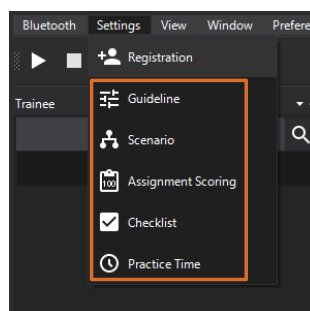


8-3-1. Guideline, Scenario, Assignment scoring, Checklist, Practice time can be customized according to the trainer's own curriculum.




- The setting values are readily reflected to the training, after you change the settings.

8-4. Match trainee's information

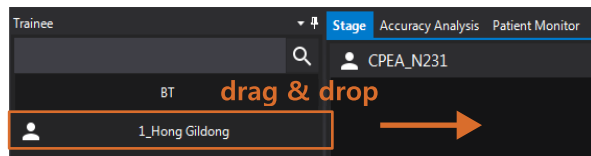

<Before matching>

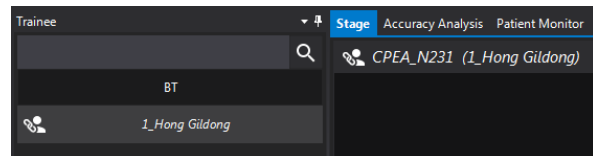


<After matching>

8-4-1. Match the registered trainee information with the manikin.



- Drag & Drop the registered trainee information to the manikin at the windows and start the practice.
- How to release matching : By double clicking the trainees info from the list will release and unload the trainee from the manikin.
- It is possible to find the previous information by clicking the trainee's name from the trainee list.
- It is possible to matching the trainee's information regardless before, after and during the training in real-time.


8-5. Practice

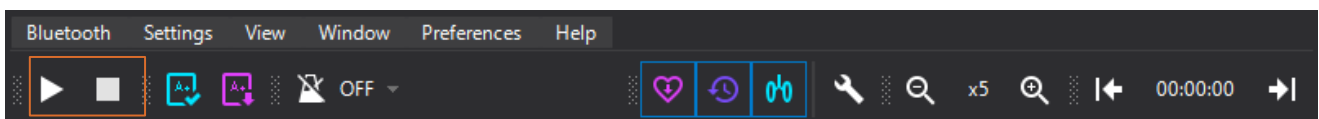


8-5-1. Start practice



- Training can be operated by using 'Start', 'Pause' 'End' icons at the top of the screen.

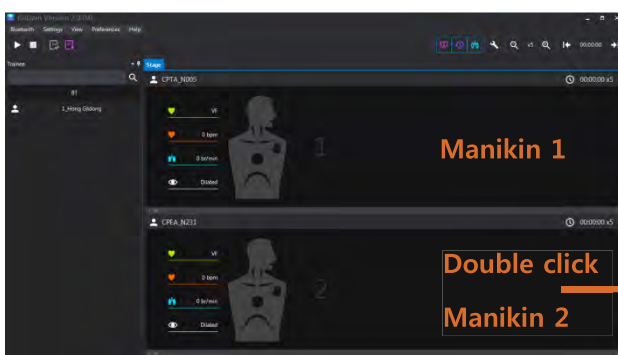

<Multi-screen with 2 manikins >

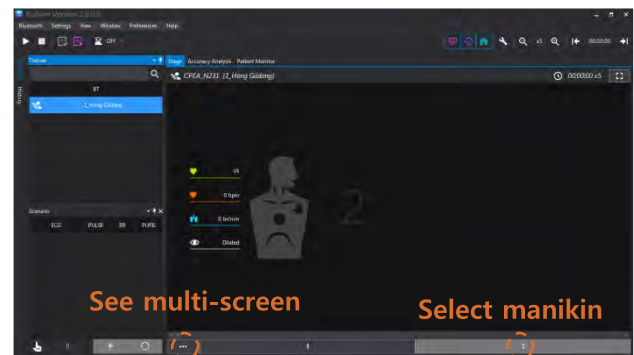


<Full screen>



- The layout of screen can be set by 'View > Replacement' menu.
- Full screen mode : Double-click the screen of related manikin to see it as a full screen.


8-6. Scenario

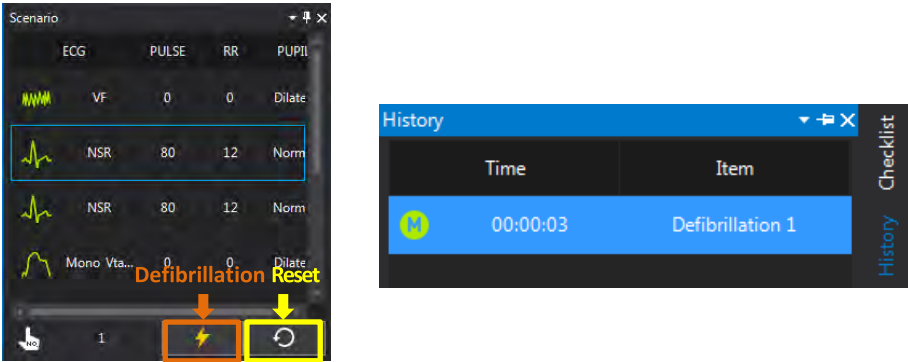

8-6-1. Defibrillation

- ‘Scenario’ is activated when full screen mode, it is possible to do defibrillation by clicking ‘Defibrillation’ icon.
- When defibrillation, the number and the time of defibrillation will be applied to ‘Scenarios’ and ‘History’.
- Scenario will return to the initial state when ‘Reset’ icon is clicked.

8-7. Patient monitor

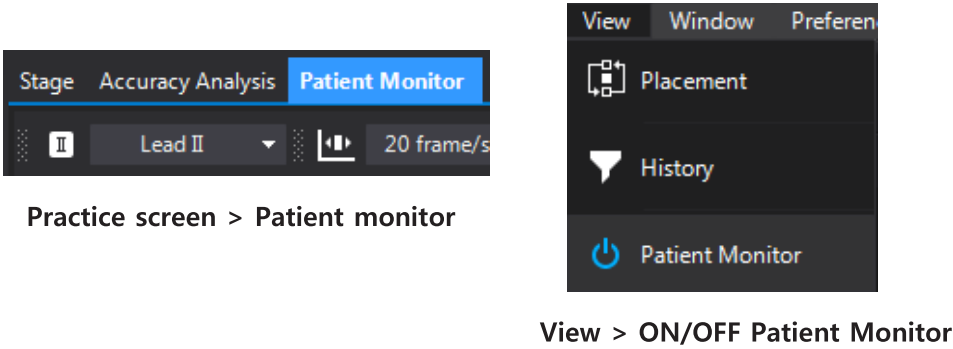

Patient monitor icon

StageAccuracy AnalysisPatient Monitor

II

Lead II

20 frame/sec


x1



50

Description	
 Select ECG Lead I , II, III	 On/Off ECG wave
 Select frame rate of ECG	 On/Off Pulse rate
 Select Y-axis scale for ECG wave	 On/Off Respiration rate
	 On/Off Number of Defibrillation
	 Select volume for HR

Activate patient monitor

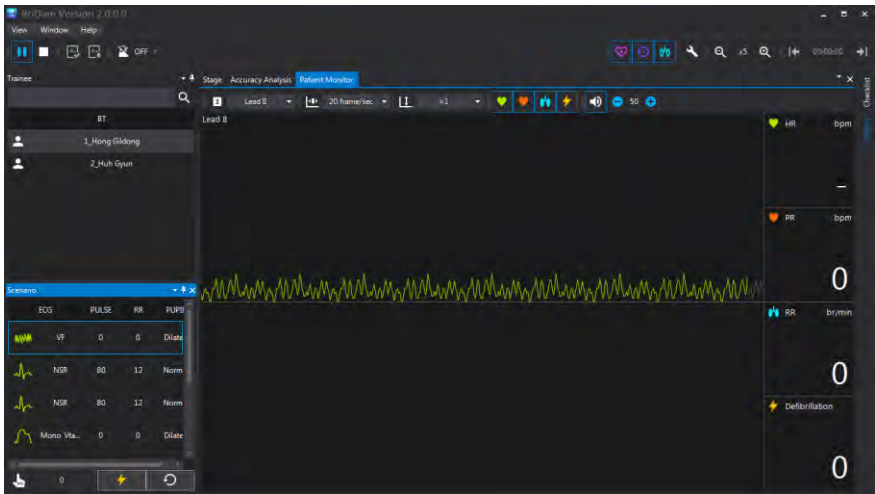

<Patient monitor screen>




Settings for Patient monitor

ECG, PR, RR, Defib information will be reflected according to scenario

Defibrillation


< When Defibrillation >

Analyze wave form

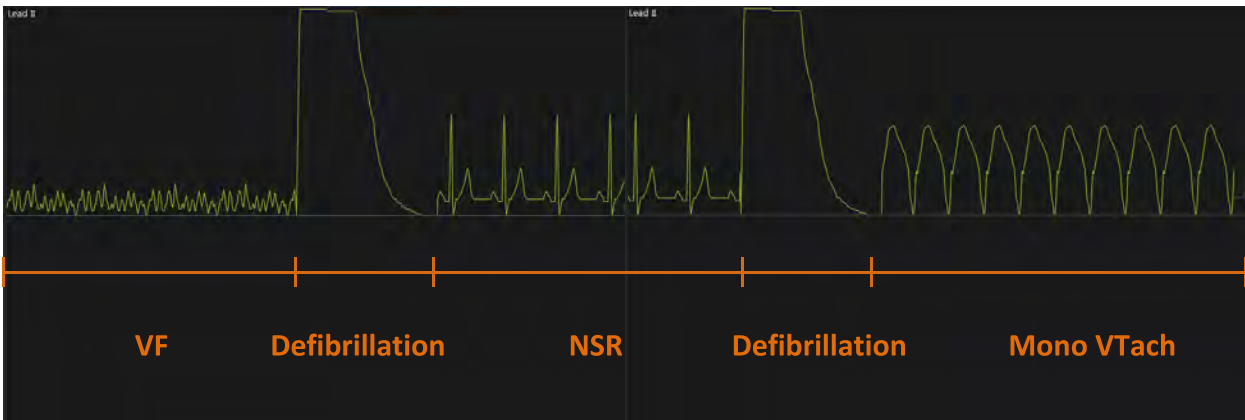

Patient status in scenario is changed according to defibrillation.


8-8. Metronome

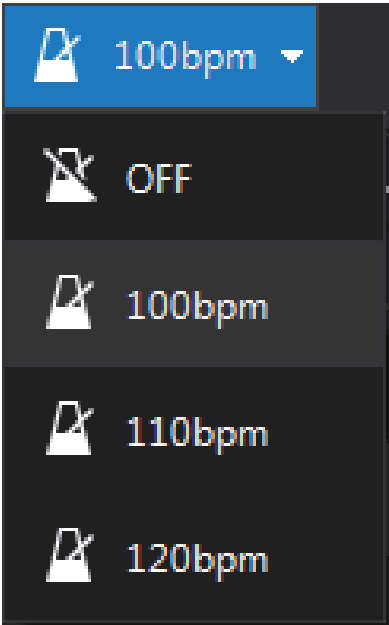

8-8-1. Set metronome

-  - Metronome setting is available in full screen mode.
- Trainee can practice chest compression with metronome sound from manikin.

8-9. Checklist

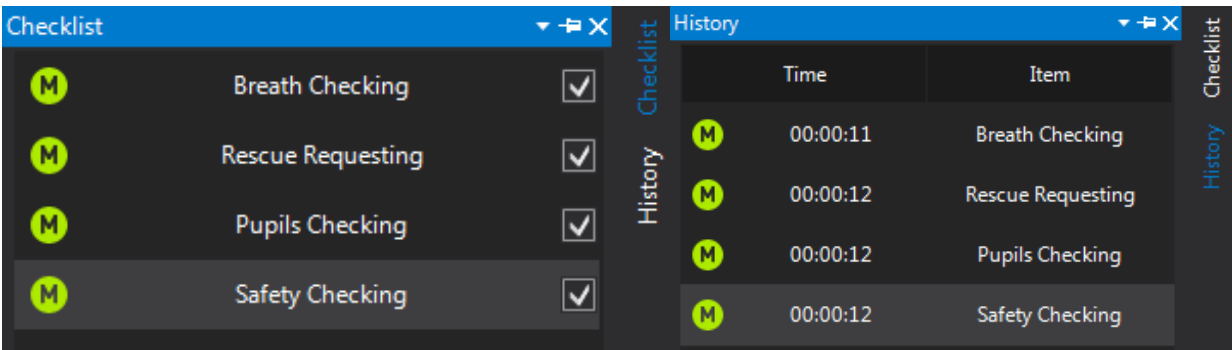


8-9-1. Check items on the list


-  - 'Checklist' is activated in the full screen mode. When items are checked, it will be reflected to the 'History' and 'Scenario' data.

8-10. Accuracy Analysis

Reference line

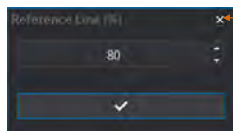

Full accuracy analysis view

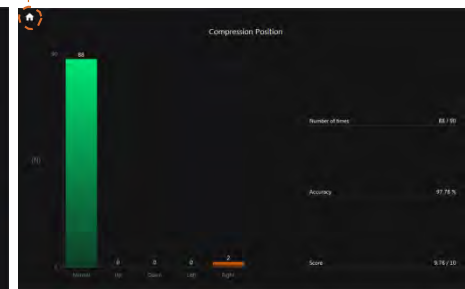


8-10-1. Check status of training with an analyzed chart



- Compression & respiration can be checked by a chart and more detailed information can be given by double clicking the bar chart.
- User can change the cut-off line by changing the Base line at the top left side of 'Full accuracy analysis' windows.


8-11. Evaluation result

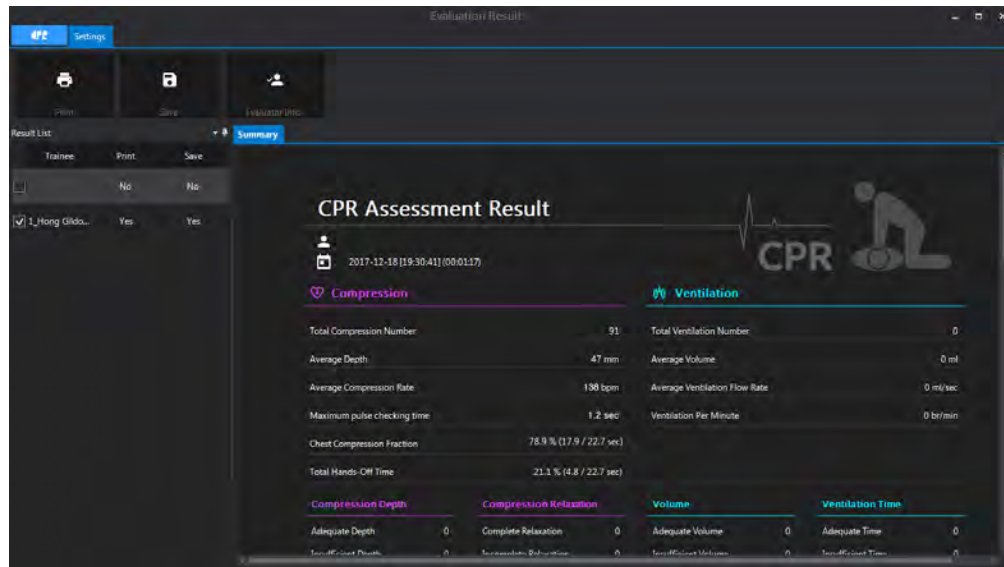

8-11-1. Check, print, save Evaluation result



- After training, user can check the result at 'Result'
- User can print & save the result at the 'Result list' which they want at the 'Result list'.
- **If the trainee information is not matched with the manikin it is not possible to save the data.**

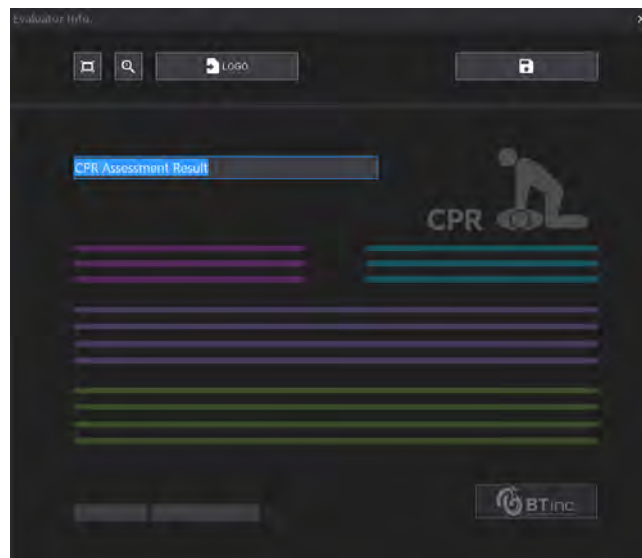


8-11-2. Set a evaluator information




- Before print and save the result data, user can set the evaluator's information to be reflected in the result data.
- 640x180 pixels is recommended for logo image.

8-12. Previous result

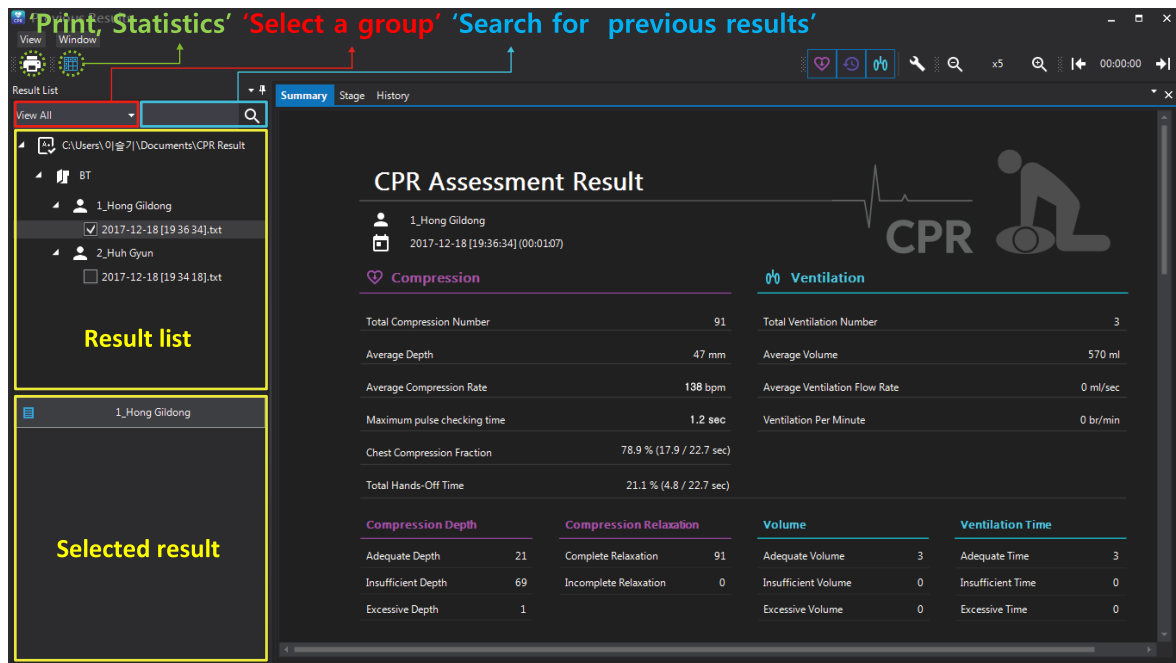

8-12-1. Analysis and print the saved previous training data

- 'Previous result' can be checked at 'Training screen > Previous results' menu.
- Selecting a group or searching for a previous results are possible at result list.
- Saved previous result is saved at 'File route' > 'Group' > 'Trainee info' > 'Previous result info.' (Training date & time)
- Selected 'Previous result' will be shown on right side of the screen and when you click 'Previous result info', the screen will be changed to desired result screen.

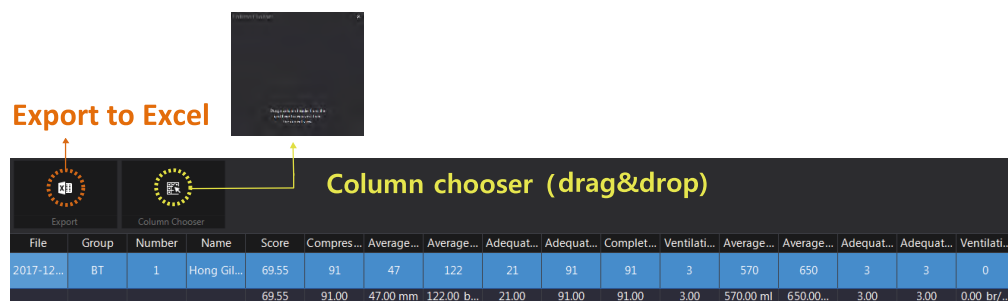

<Statistics>

8-12-2. Check and print the statistics of result

- User can check the statistics of multiple training data at the 'Previous results > Statistics'.
- User can change the order of displayed data through drag & drop the each items.
- The item can be added or deleted through item drag & drop at the top of the item selector.
- User can forward the statistics data to excel file when they click the 'Export' button.

9. Convenient functions

9-1. Register trainee information

- It is convenient edit since it is possible to 'copy/paste' 'trainee information'.
- Users can set a file path for result files.

9-2. Set scenario

- Pulse rate, respiration rate and pupil reaction is changes according to ECG rhythm and some items can be adjusted separately.

9-3. Set checklist

- Setting with negative number is available to subtract points.

9-4. Delete previous result

- To delete the data, place the mouse cursor on the name of the item on the list and 'Right-click the mouse > delete'.