


Intradermal Injection Model

BT-CSID

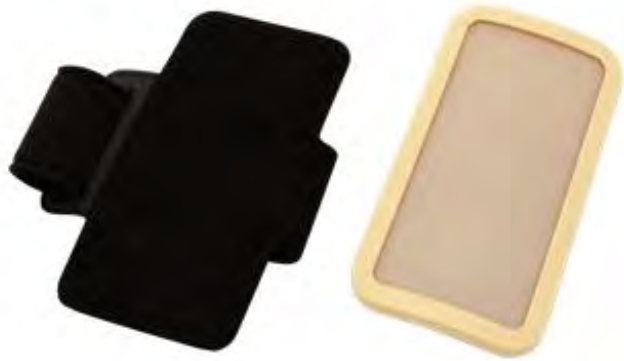

- **This model is a training device for intradermal injection practice.**
Do not use it for different purposes beside injection practice.

1. Features

- ① Feeling and resistance of injection interface are similar to real skin.
- ② Practice diagnostic test, tuberculin test, and vaccination are possible.
- ③ Realistic practice is possible through wearing it to mock patient.
- ④ Easy installation and handling.

2. Components

- ① Intradermal injection model – 1EA (82x152x20mm)
- ② Mounted arm band – 1EA (340x150x3mm)

3. User guides

- ① Wear the arm band to the arm.
- ② Attach the intradermal injection model to the arm band.
- ③ Practice with a syringe.
- ④ Use clean water only.
✖ **It might cause life-shortening to use injection solution beside water.**
- ⑤ After practice, storage the product following the '4. Maintenance and storage.'

4. Maintenance and storage

- ① Please wipe the product with **ethanol cotton or fabric**.
- ② Please putting the **Baby powder** all around the product and wipe the product again with your hands or soft clothes.

5. Caution

* Read it carefully to avoid damaging the product and injuring yourself.

Pay attention to the following series of warnings.

- ① Do not use it for different purposes beside intradermal injection practice.
- ② User can't take warranty service if user disassemble, repair, or modify this product on your own. Also, anyone who remodels or copies it for commercial purpose can be subjected to legal penalty.
- ③ Do not keep the product near hot places or cold places.
- ④ Do not tighten arm band too much.
- ⑤ BT Inc. can change or modify the product specification without any notice in order to improve the product.

