

A woman and a mannequin are lying on a wooden floor, surrounded by red rose petals. The woman, on the left, has visible blood on her face and hands and is holding a rose. The mannequin, on the right, has colorful hair and is also holding a rose. The scene is dimly lit, creating a somber and urgent atmosphere.

Be
READY

DANi Teen & Young Adult Advanced Training Solution

Nasco
HEALTHCARE

Quick Start Guide

DANi is packaged with arms and female genitalia attached. Legs have been disconnected to prevent possible damage during shipping.

1. Carefully unpack the product's individual components (see Fig. 1).

Your DANi manikin includes:

- Head
- Torso
- Arms
- Legs
- Female genitalia
- Male genitalia
- Accessories



Figure 1

2. Connect left and right legs to the torso (see Fig. 2).
3. Female genitalia attached, or connect male genitalia (see Fig 3).
4. Your product is ready for use.

DANi comes pre-assembled to support nasogastric tube procedures (lavage, gavage). Please see the Set Up section of the instruction manual if you want to configure your product to support different airway management procedures.

For more information on your DANi manikin, please contact our customer service team at 1-833-NASCOHC (627-2642) or email us at info@nascohealthcare.com. We are happy to help you!



Figure 2



Figure 3



Please visit
<https://nhcpartnerportal.com/products/manuals/>
for complete instructions.



Nasco Healthcare
16 Simulaids Drive
Saugerties, NY 12477
1-833-NASCOHC (627-2642)
info@nascohealthcare.com
www.nascohealthcare.com

©2023 Nasco Healthcare

PP9201

PC 101037 01/23