


INSTRUCTION MANUAL

Enema Administration Simulator

LF00957

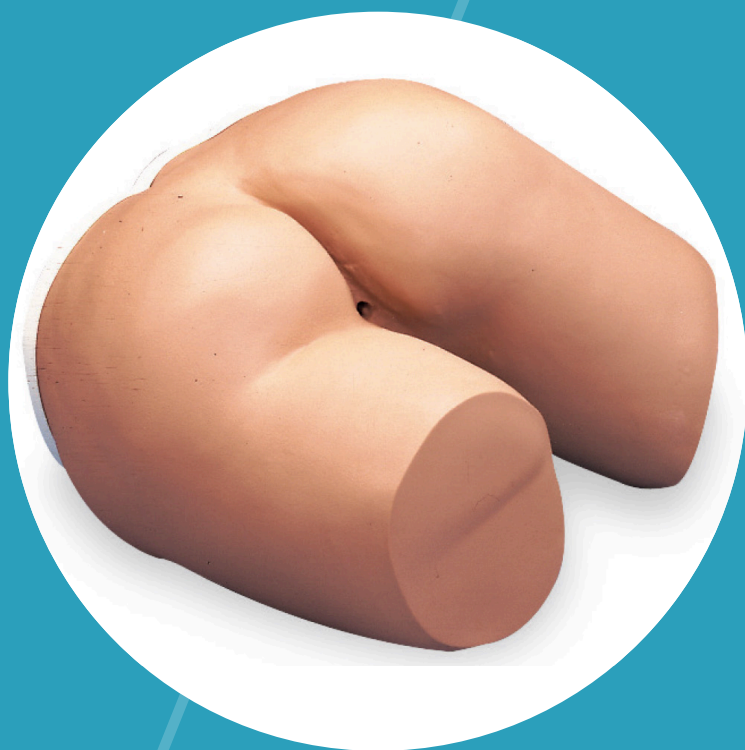

5 Year Warranty

Nasco
HEALTHCARE

About the Simulator



The Enema Administration Simulator is designed to demonstrate and practice enema procedures. Standard apparatus can be used to introduce fluids. This simulator must be treated like a live patient or damage will result.

LIST OF COMPONENTS

1. Enema Bag
2. Latex Glove
3. Hevi-Lube Lubricant
4. 18" Vinyl Tube
5. Carrying Case

INTERNAL STRUCTURE

All aspects of the simulator are life-like in every detail, and the action of the unit is extremely realistic. The upper buttock is flexible so that the trainee must raise the buttock in order to locate and make insertion into the anus. Simulated bowel will accommodate the introduction of fluids.

GENERAL INSTRUCTIONS FOR USE



Set-Up Procedure

1. Fill flexible Enema Administrator Unit with clear water.
2. Hang a bag a maximum of 18" above the simulator (**See Figure 1**).

3. Prior to each use of the simulator, the anal orifice and enema catheter **MUST** be lubricated with Hevi-Lube Lubricant (LF01050). Do not use substitutions. **CAUTION:** Do not use large amounts of Hevi-Lube. Over application may block drain tube.
4. The drain tube, which is stored in the case, must be attached to the connector on top of the simulator before the introduction of fluid. Place the drain tube in a sink, pan, or bucket (not included) for continuous drainage of administered fluids (**See Figure 2**).

Procedures that can be performed on this simulator:

The standard enema procedures can be performed using conventional enema apparatus. No special equipment is required.

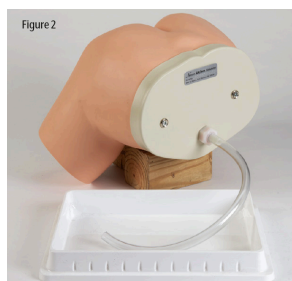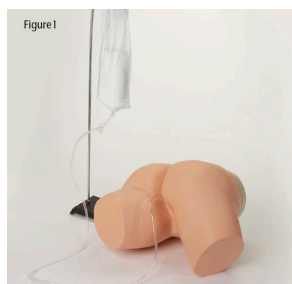

Care of the Simulator

Normal soil can be removed with mild soap and lukewarm water. Use Cleaner (LF09919) to remove stubborn stains. Spray soiled area with cleaner and wipe clean with soft cloth or paper towel. Before storage, drain completely. Hold simulator upside down and drain all water. Remove 18" drain tubing. Dry all surfaces.

CAUTIONS

Solvents or corrosive materials will damage the simulator. Never place simulator on any kind of printed paper or plastic. These materials will transfer an indelible stain. Ballpoint pens will also make a permanent stain.

SUPPLIES/ REPLACEMENT PARTS

LF09919 Cleaner

LF01050 Hevi-Lube

LF01024 18" Drain Tubing

LF01098 Fluid Administration Set



Nasco Healthcare
16 Simulaids Drive
Saugerties, NY 12477
1-833-NASCOHC (627-2642)
info@nascohealthcare.com
www.nascohealthcare.com