

INSTRUCTION MANUAL

# Male Catheter Model

LF01150



*Nasco*  
HEALTHCARE


# Using the Model

## Using the Model

- Use this model to practice sterile technique and proper cleansing procedures before, during, and after insertion.
- The model will accept any catheter size 16 or smaller.
- Be sure the balloon on the catheter is fully deflated before insertion or removal. Always lubricate the catheter thoroughly before each insertion.
- Lift the penis to straighten the urethra. Gently insert the catheter into the urethral meatus, through the prostate and into the bladder. Proper insertion in a live patient is evidenced by a flow of urine through the tube.



# Caring for the Model

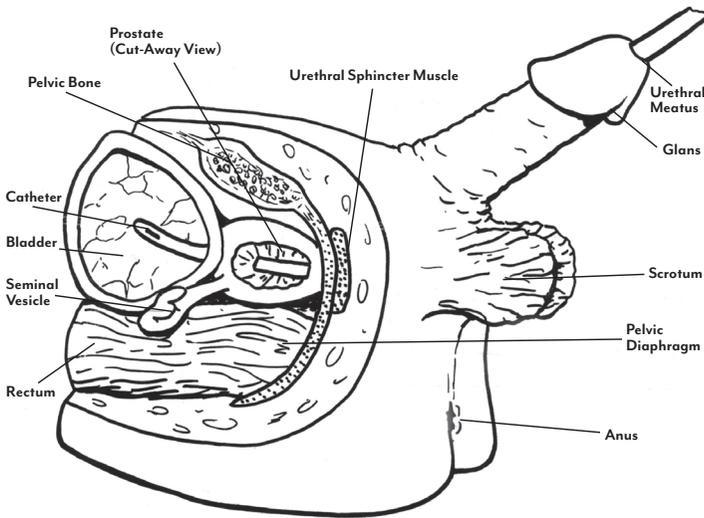

## Caring for the Model

- The model may be washed in mild soap and water if it becomes soiled.
- Always store it in the plastic bag when not in use.
- Printed material or ballpoint pens will transfer indelible stains to the model. Certain finishes may also be damaged from prolonged contact with the model.

## AVAILABLE SUPPLIES

- LF01127** 16 FR Foley Catheter
- LF01128** 16 FR Foley Catheter, 10-pack
- LF00985** Lubricant Kit, 2 oz. bottles, 6-pack



**Nasco Healthcare**  
16 Simulaids Drive  
Saugerties, NY 12477  
1-833-NASCOHC (627-2642)  
info@nascohealthcare.com  
www.nascohealthcare.com