


Instruction Guide for EVA Pelvic Teaching Model No. 1900

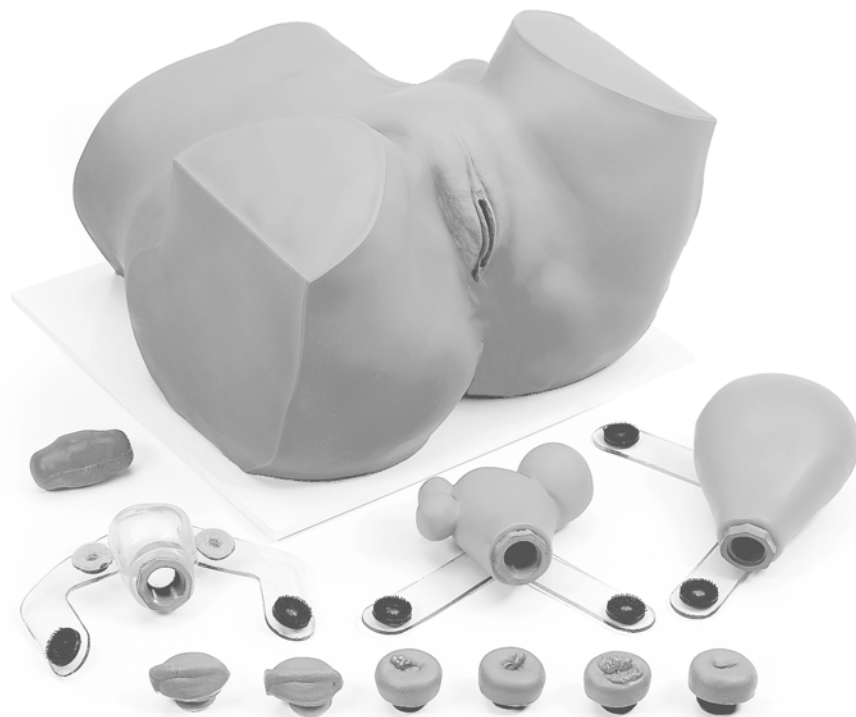

 **Simulaids**

Thank you for buying an American made product.

The EVA Pelvic Teaching Model is a life-like pelvis with a variety of anatomical parts. This model is designed to facilitate the teaching, demonstration, and practice of IUD insertion and removal as well as assisting students in performing a pelvic examination. The EVA model permits students to practice a variety of family planning and gynecological procedures including:

Abdominal palpation

Speculum insertion and removal

IUD insertion and removal (includes tenaculum placement on cervix and uterine sounding)

Bimanual and rectovaginal examination for:

Comparative evaluation of the size, shape and position of a normal, enlarged (pregnant) and fibroid uteri

Identification of an adnexal or fecal mass

Identification of normal and abnormal cervixes (polyp and carcinoma)

Materials:

Normal cervix (for IUD insertion/removal) (1)

Normal parous cervix (1)

Cervix with ectopic characteristics (1)

Cervix with endocervical polyp (1)

Cervix with neoplasia (carcinoma) (1)

Normal uterus (transparent for IUD insertion/removal) (1)

Pregnant uterus (10 week size) (1)

Fecal mass (1)

Uterus with Leiomyomata (fibroids) (1)

Adnexal Mass (2)

The cervixes and all uteri are equipped with a locking feature for use in the model. To attach these components follow these simple steps:

Place cervix through hole from inside the vagina

Wrap tabs around uterus and hold in place with a rubber band

From the back of the manikin place fingers under the end of the vagina pushing against cervix

Thread the uterus onto the cervix then remove the rubber band

Attach the Velcro tabs to the side walls of the pelvis making sure Velcro dots are aligned

To disengage the uteri follow these steps:

Release the Velcro tabs

Wrap tabs around uterus and rubber band in place

Grasp the cervix and uterus

Rotate the uterus until completely unthreaded

Care and Maintenance:

Your EVA pelvic model has been constructed using materials that approximate skin texture. Care should be taken by the user so as not to damage the unit.

When using the EVA model always employ the same gentle techniques that would normally be used when working with a human.

Any instruments or fingers used in the examination process should be well lubricated. Lubricating jelly or a mild soap solution should be adequate and will not harm the model.

Any cleaning of the unit should be done using only a mild detergent solution and rinsing with clean water.

NOTE: DO NOT STORE PRODUCT IN PLASTIC WRAP OR PLASTIC BAGS AS THIS MAY CAUSE THE MODEL TO DISCOLOR.

Return Policy: Should it be necessary to return an item for any reason contact our Customer Service Department to obtain an RGA Number. Please refer to your invoice number when phoning in your request for returning merchandise. Should you have any questions or wish further information on any product we manufacture call or write our Customer Service Department.

Limited Three (3) Year Warranty: Simulaids warrants this product to be free from any defect in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date of invoice when the product was shipped to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product. Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory. At no time will any product be accepted at the plant without proper return authorization issued by Simulaids.

Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Any product considered for warranty work must be identified by serial number and invoice number from the agency through whom the product was purchased. Without this information the product will not receive a return authorization number as required above.



 **Simulaids**



PO Box 1289-16 Simulaids Dr-Saugerties NY 12477

845-679-2475 or 800-431-4310

Fax 845-679-8996

www.simulaids.com

e-mail: info@simulaids.com