




Forceps/Vacuum Delivery OB Manikin

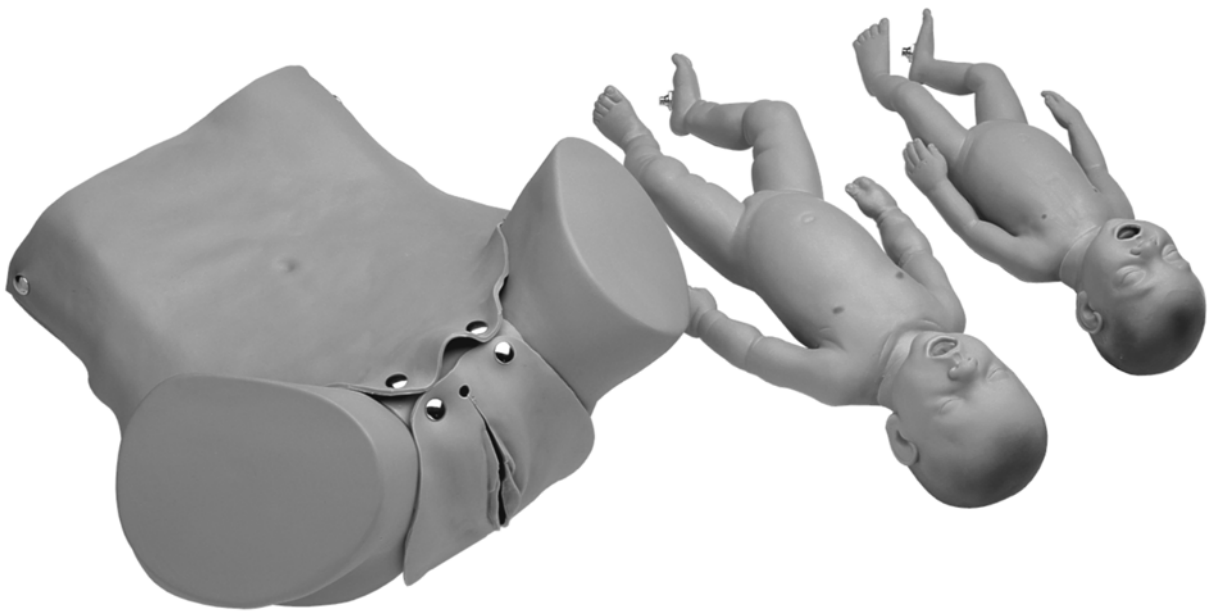

INSTRUCTIONS FOR USE, CARE, AND MAINTENANCE

Forceps Manikin Baby Instructions

The two Baby Fetuses are air-filled manikins with water filling capability built to provide a more life like weight distribution.

To fill with air:

1. Slip tubing from air pump over metal valve on manikin's left foot.
2. Turn to open.
3. Pump bulb until desired firmness is achieved.
4. Screw valve tight to reseal.
5. Remove bulb assembly.

Note: On larger baby, do not over inflate. The baby will be more flexible with less air.

To fill with water:

1. Hold manikin upside down.
2. Open both metal valves located on the bottom of each foot.
3. Attach hose from faucet to metal valve on the bottom of left foot.
4. Fill with warm water until water begins to flow from the valve on the right foot.
5. Shut off flow of water.
6. Remove hose from valve on left foot.
7. Gently squeeze air from manikin until water flows from left valve.
8. Hold and close valve tightly.

To drain water:

1. Place manikin's feet into sink and open both metal valves.
2. Let drain 15 minutes or until all water has drained out.
3. Inflate with air for storage.
4. After use, the manikin should be washed with warm soapy water, rinsed well, dried, and lightly dusted with baby powder.

Water must be drained from manikin every seven (7) days to prevent microbiologic growth inside the manikin. Inflate manikin with air for storing.

Important Notes: To reduce friction between the pelvis and the fetus during delivery, spread baby powder on all of the surfaces that come into contact with each other. The manikin will stay in place on a smooth surface if you first moisten the bottom of the suction cups before securing the unit to the table. Always remove the baby from the birth canal at the end of every class. Facilitate a non-moving manikin by wetting the suction cups on the bottom of the pelvis before placing on a work surface.

No. 1377 Forceps / Vacuum Delivery Manikin
Sh. Wt. 15 lbs. Size 21" x 13" x 17"

Replacement Parts:

171 Opaque Overlay

172 Premie Manikin

173 Newborn Manikin

182 Vulva

162 Methyl Cellulose (simulated amniotic fluid) 225

Simulated Blood Powder (makes 1 gallon)

RETURN POLICY

Should it be necessary to return an item for credit for any reason, contact our Customer Service Department to obtain an RGA Number. Please refer to your invoice number when phoning in your request for returning merchandise. Should you have any questions, or wish further information on any product we manufacture, call or write our Customer Service Department.

Simulaids' Limited Three (3) Year Warranty

Simulaids warrants this product to be free from any defect in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date of invoice when the product was shipped to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product. Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory. At no time will any product be accepted at the plant without proper return authorization issued by Simulaids.

Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Any product considered for warranty work must be identified by serial number and invoice number from the agency through whom the product was purchased. Without this information the product will not receive a return authorization number as required above.



Manufacturers of Training Manikins, Casualty Simulation Kits,
Medical Training Devices

PO Box 1289 - Saugerties NY 12477

(845)-679-2475

Toll Free: (800)-431-4310 Fax: (845)-679-8996

www.simulaids.com