



Surgical Bandaging Simulator

“Surgical Sally”

LF00929U

INSTRUCTION MANUAL



5-Year Warranty

ABOUT THE SIMULATOR

The **Life/form®** “Surgical Sally” Bandaging Simulator is a replica of a 29" x 17" female with 14 surgical wounds, including a mid-sternal split with chest tube drain, a stage 2 sacral pressure injury, and a leg amputation stump. The skin has been reformulated so it is drier, allowing bandages to adhere better. Wound closures have been updated with surgical staples for the mid-sternal split, thoracotomy, nephrectomy, laparotomy, abdominal hysterectomy, and amputation stump.



Actual product may vary slightly from photo. Nasco reserves the right to change product color, materials, supplies, or function as needed.

FEATURES

The simulator features the following simulated wounds:

- Thyroidectomy
- Mid-sternal split with chest tube drain
- Mastectomy with simulated drain
- Cholecystectomy with simulated T-tube
- Laparotomy
- Appendectomy
- Colostomy
- Ileostomy
- Abdominal hysterectomy
- Thoracotomy
- Nephrectomy
- Laminectomy
- Stage 2 sacral pressure injury
- Leg amputation stump

The **Life/form®** “Surgical Sally” Bandaging Simulator comes complete with a durable carrying case.

CAUTIONS

1. Never place the simulator on any kind of printed paper or plastic. These materials will transfer indelible stains. Ballpoint pens will also make indelible stains.
2. Do not use solvents or corrosive materials to remove residual adhesives. Use only Glue-Off (LF01023U) or warranty is void.
3. Do not use povidone-iodone prep solutions. These solutions will indelibly stain the simulator.

CARE OF SIMULATOR

Normal soil can be removed with mild soap and water. Nasco Cleaner (LF09919U) will remove stubborn stains from simulator. Simply apply Nasco Cleaner to soiled area and wipe clean with a soft cloth or paper towel.

AVAILABLE SUPPLIES

LF01023U Glue-Off

LF09919U Nasco Cleaner

Other Available **Life/form** Simulators

- LF01005U** First Aid Arm
- LF01028U** Suture Practice Arm
- LF01034U** Suture Practice Leg
- LF01038U** Episiotomy Suturing Set
- LF01042U** Suture Kit
- LF01063U** Stump Bandaging, Upper
- LF01064U** Stump Bandaging, Lower
- LF01083U** Tracheostomy Care



LF01005U



LF01028U



LF01042U



LF01063U



LF01064U



ONLINE:
EMAIL:
PHONE:
ADDRESS:

NascoHealthcare.com
lifeform@enasco.com
1.800.431.4310
901 Janesville Ave.
Fort Atkinson, WI 53538