


INSTRUCTION MANUAL


Suture & Stapling Practice Arm

LF01028 – Light
LF00670 – Medium
LF00673 – Dark

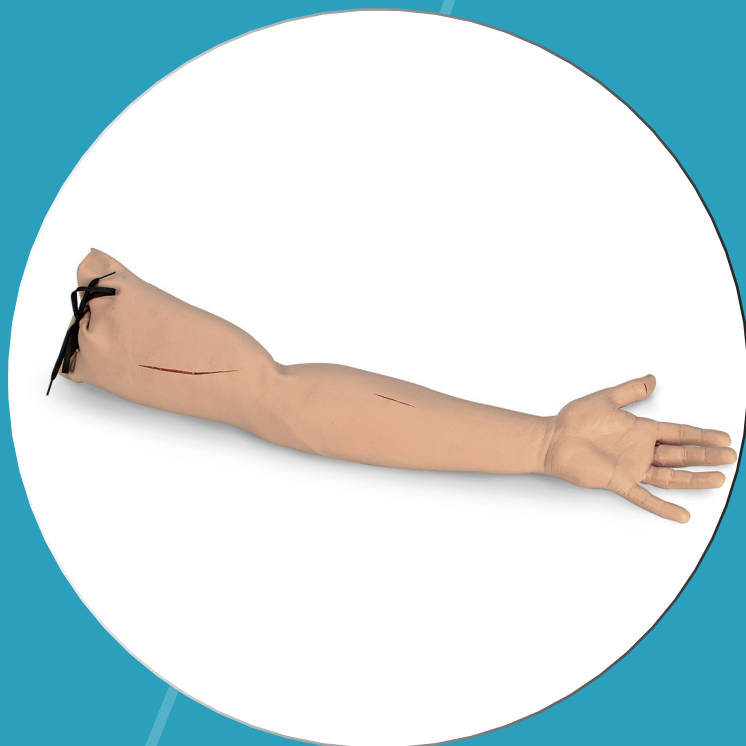

Nasco
HEALTHCARE

About the Simulator

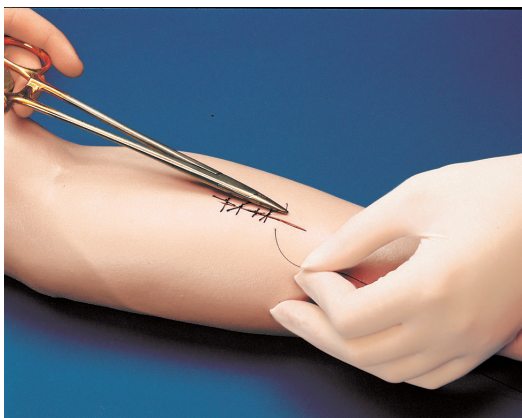

This practice and demonstration suture arm should be considered disposable, but with proper care and treatment, it will serve its purpose for a long period of time. The three “wounds” provided should be considered only samples of what can be done. With a sharp scalpel, additional “wounds” can be cut at any desired depth or location. Each can be repeatedly sutured until the skin at any one site becomes so heavily perforated that it will no longer support more suturing.

The “wounds” can be bandaged with most standard adhesive bandages. Residual adhesive can be removed with Glue-Off (LF01023).

Care Of Simulator

Do not use providone-iodine prep solutions. These solutions will indelibly stain the arm. Normal soil accumulated on the surface of the simulator can be removed with mild soap and lukewarm water. Use Nasco Cleaner (LF09919) to remove stubborn stains from simulator. Simply spray soiled area and wipe clean with a soft cloth or paper towels. When not in continuous use, it is recommended to store the simulator in the storage box furnished, or cover it with a cloth or sheet of plastic.

Cautions

Solvents or corrosive materials will damage the simulator. Never place simulator on any kind of printed paper or plastic. These materials will transfer indelible stains. Ballpoint pens will also make indelible stains.



Supplies/Replacement Parts

LF01011 Suture Tool Kit

LF09919 Nasco Cleaner

Other Available *Life/form* Simulators

LF00890U Interactive Suture Trainer, Light

LF00892U Interactive Suture Trainer, Dark

LF00929U Surgical Bandaging

LF01005U First Aid Arm

LF01034U Suture Practice Leg

LF01042U Suture Kit

LF01046U Facial Suturing Module Set


LF01047U Pediatric Suture Head

LF01063U Stump Bandaging, Upper

LF01064U Stump Bandaging, Lower



LF01034U

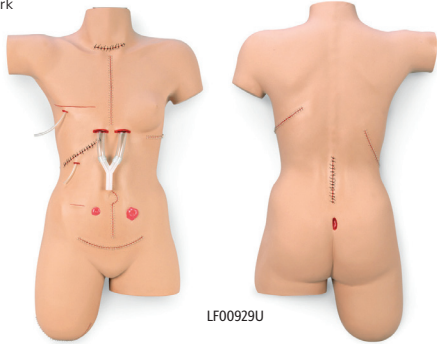

Front

Back



LF01046U



LF00890U



LF01047U

Nasco Healthcare
16 Simulaids Drive
Saugerties, NY 12477
1-833-NASCOHC (627-2642)
info@nascohealthcare.com
www.nascohealthcare.com