


XR-Clinic Pico 4 setup manual

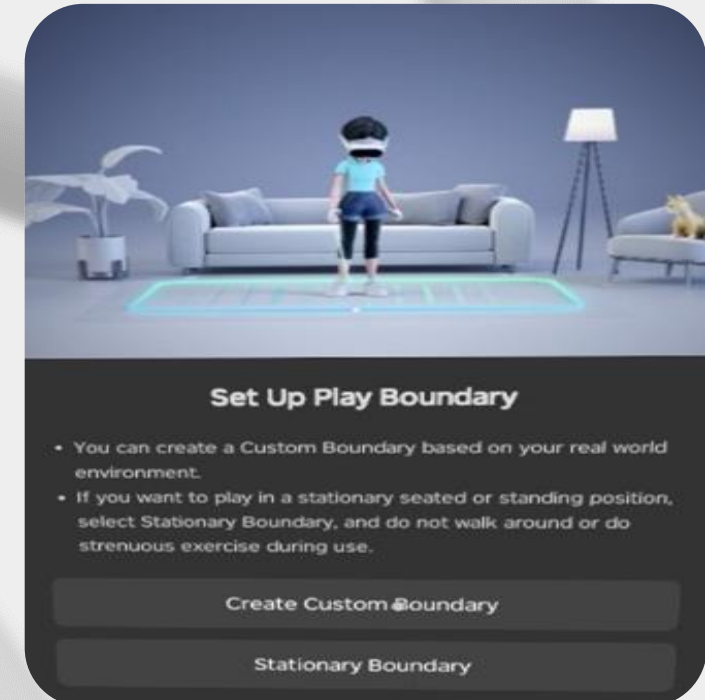



Set up the headset

- Charge the headset using USB cable
- Hold the power button on the right side to turn on VR headset
 - Create the boundary or chose the stationary one
- Connect the headset to WI-FI and install system updates





Install the application

- Go to the Library - Not Installed
- Click on the “Beta” icon to download XR-Clinic
 - Go to the Library - Apps
- Click on the XR-Clinic application to launch it

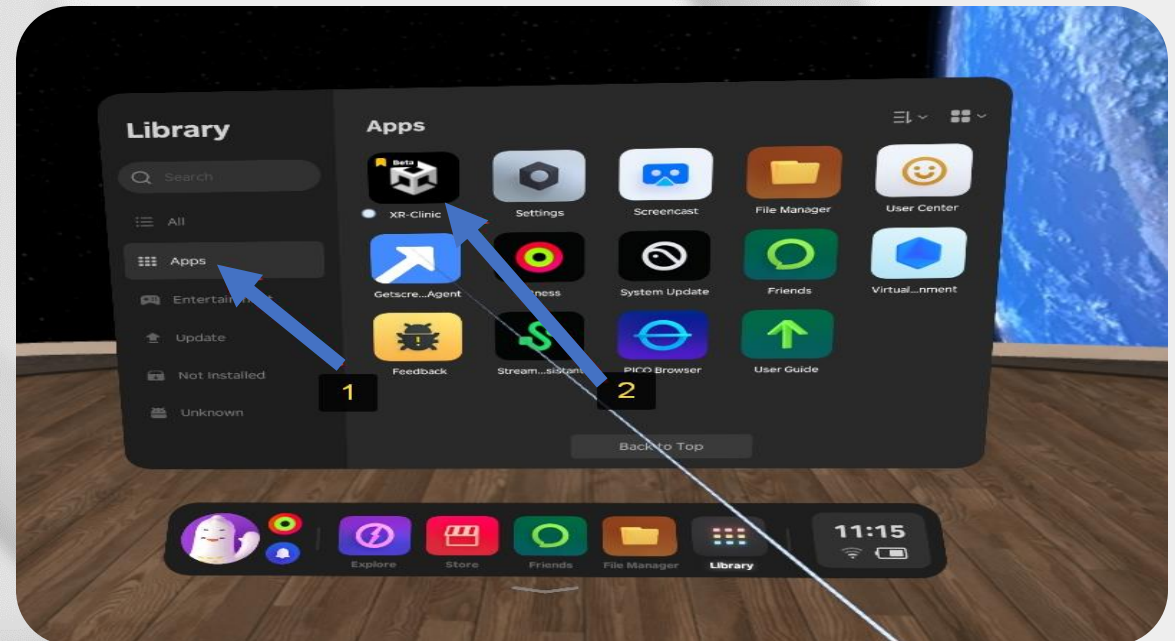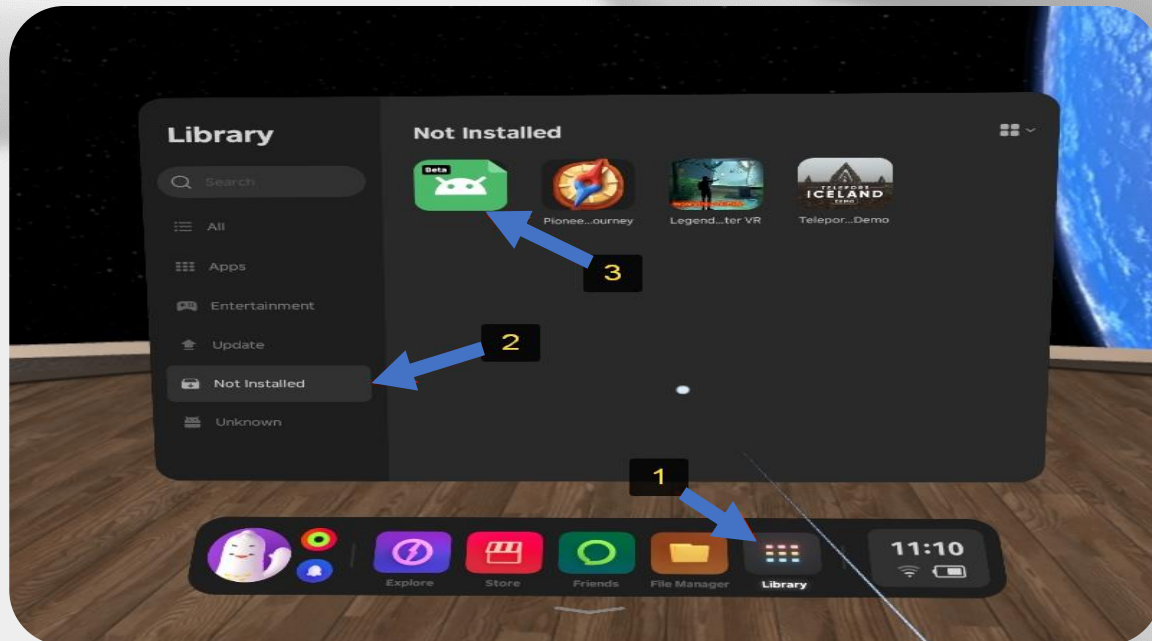



Log in the XR-Clinic Mobile

- The Profile key is provided in the welcome letter to your e-mail
- Enter the Profile key on the start screen of the XR-Clinic and click “Confirm”

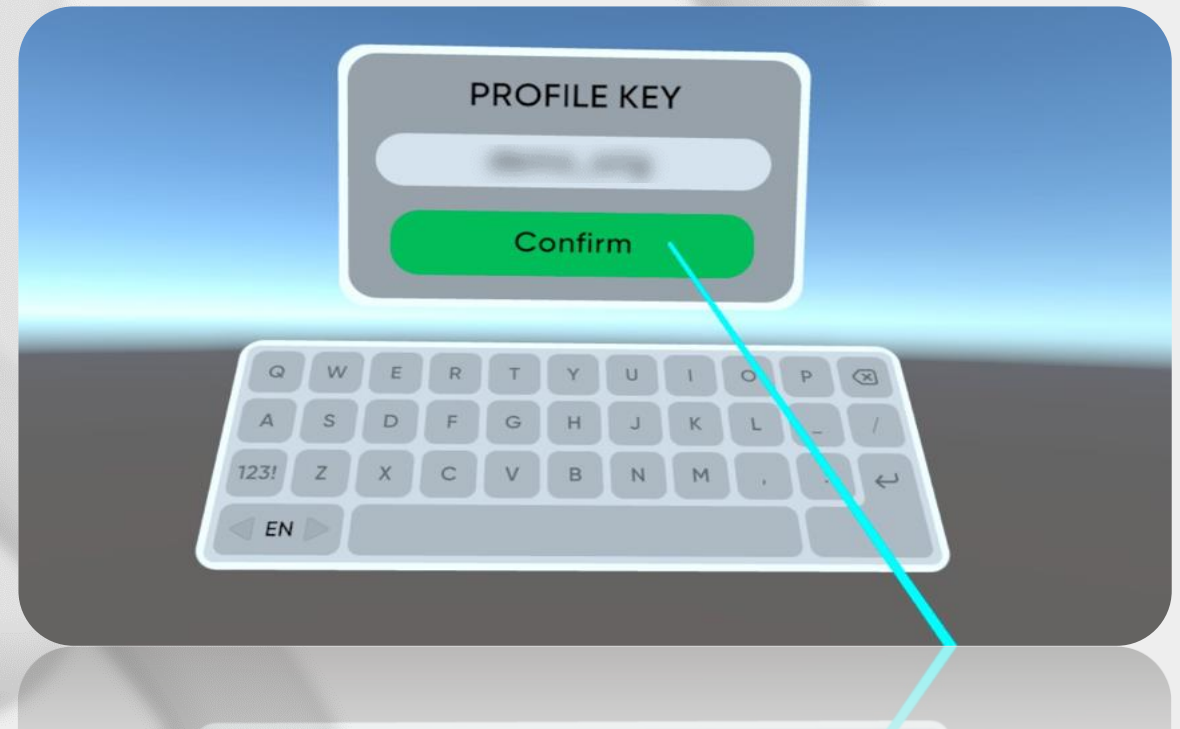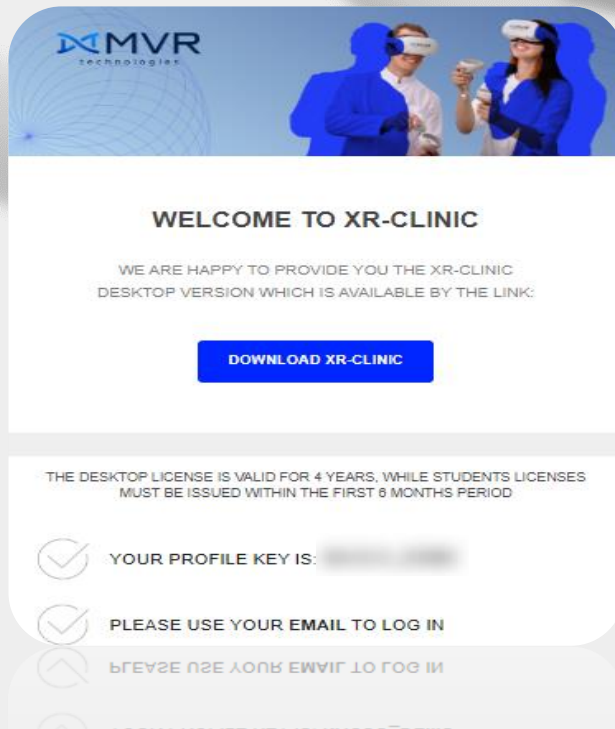

Log in the XR-Clinic Mobile

- Allow the application the access to file system (if needed)
- The license GUID should be confirmed in your Smart Center

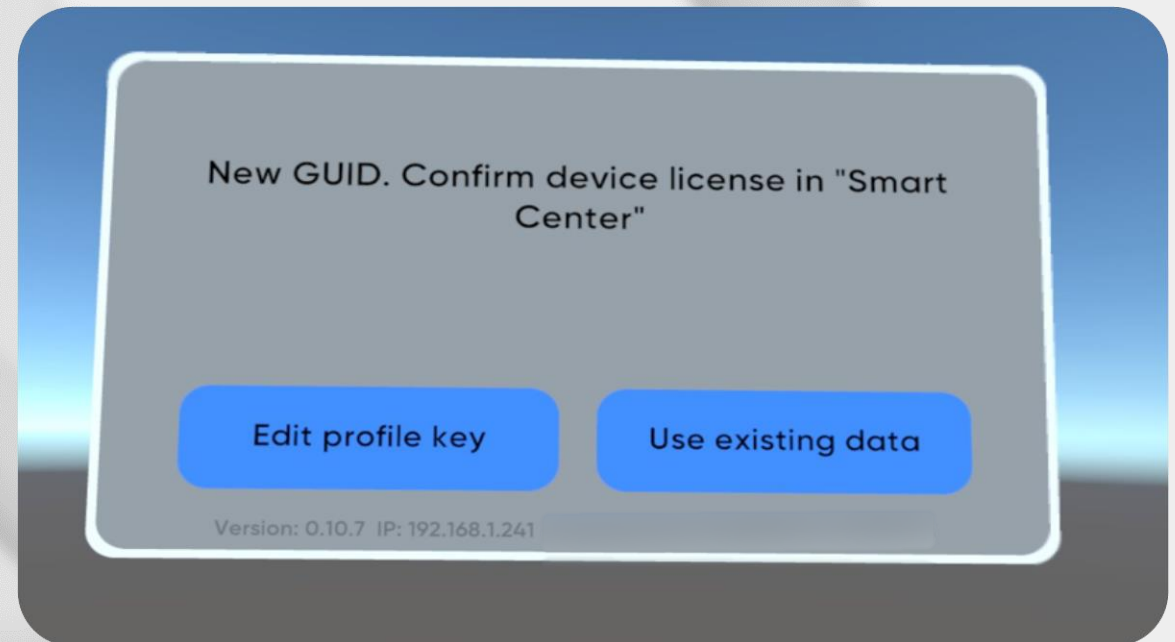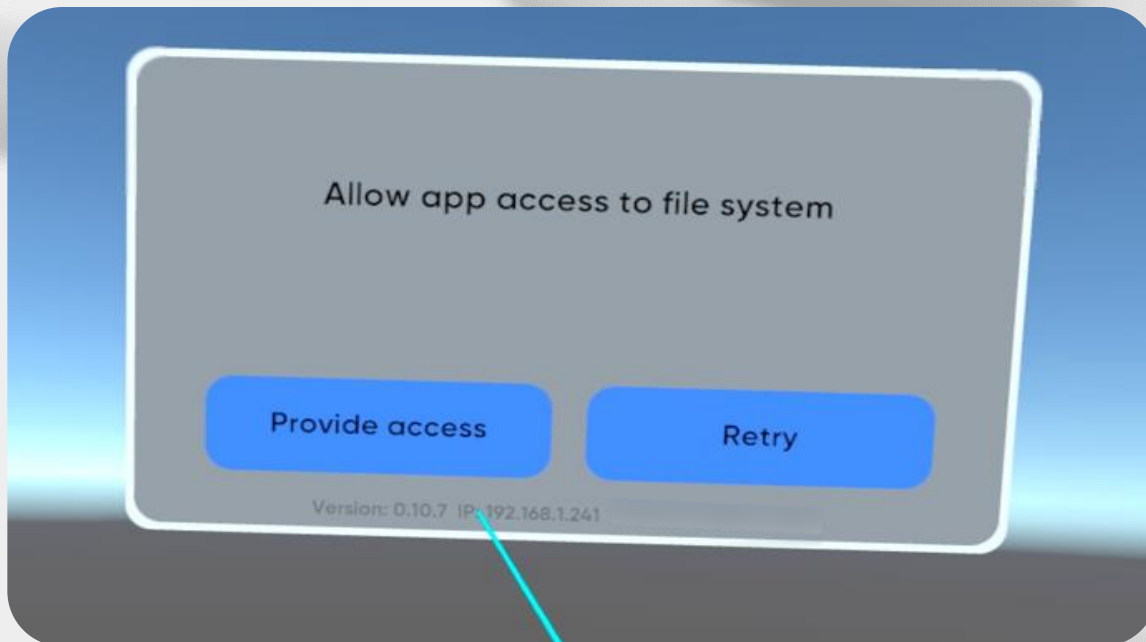

Log in Smart Center

- Go to: <https://case-manager.pro:8050>
- Enter login and password provided in the welcome letter and click “Sign in”
- Go to the “License” section

☐ Show password

☐ Remember login details.

Sign inRecover password

SMART CENTER

License Examination

Statistics

Filter

Calendar

14-10-2000

Name

Search

Group

Search

Simulator

☐ Outpatient

Clinical case

Search

Other

☐ License

☐ Without a license

☐ Desktop

☐ VR-glasses

☐ Exam

Reset

Date	Name	Group	Clinical case	Examination	License	Platform	Score	
X 2023-04-20 17:18:54	Twain Mark DemoSale		Acute appendicitis, uncomplicated	No	Yes	🖥️	5	Open
X 2023-04-19 17:41:02	Twain Mark DemoSale		Acute appendicitis, uncomplicated	No	Yes	🖥️	25	Open
X 2023-04-19 17:28:16			Acute appendicitis, uncomplicated	No	No	🖥️	40	Open
X 2023-04-19 16:27:31	Twain Mark DemoSale		Community-acquired non-serious segmental pneumonia S7 of the left lung. Respiratory failure 0st	No	Yes	🖥️	0	Open
X 2023-04-19 15:27:55	Twain Mark DemoSale		Acute appendicitis, uncomplicated	No	Yes	🖥️	0	Open
X 2023-04-19 14:19:38	Twain Mark DemoSale		Aortic insufficiency, Arterial hypertension, grade 2 Chronic heart failure, functional class 2	No	Yes	🖥️	0	Open
X 2023-04-19 14:17:46	Twain Mark DemoSale		Aortic stenosis, Chronic heart failure functional class 2	No	Yes	🖥️	3	Open
X 2023-04-19 14:16:09	Twain Mark DemoSale		Bronchial asthma persistent, moderate severity, exacerbation, uncontrolled	No	Yes	🖥️	0	Open
X 2023-04-18 22:35:24	Twain Mark DemoSale		Community-acquired non-serious segmental pneumonia S7 of the left lung. Respiratory failure 0st	No	Yes	🖥️	100	Open
X 2023-04-13 19:10:44	Twain Mark DemoSale		Gastroesophageal reflux disease, erosive esophagitis of stage A.	No	Yes	🖥️	3	Open
X 2023-04-13 19:09:13	Twain Mark DemoSale		Community-acquired non-serious segmental pneumonia S7 of the left lung.	No	Yes	🖥️	0	Open
X 2023-04-13 19:09:13	Twain Mark DemoSale		Community-acquired non-serious segmental pneumonia S7 of the left lung.	No	Yes	🖥️	0	Open
X 2023-04-13 19:09:13	Twain Mark DemoSale		Community-acquired non-serious segmental pneumonia S7 of the left lung.	No	Yes	🖥️	0	Open

Statistics

Simulator

Average score

Output

25

Confirm the Mobile license in Smart Center

- Your headset GUID is waiting for confirmation in the list of Mobile licenses
 - Click the “tick” icon to confirm the device
 - Headset GUID will appear in the list of Mobile licenses

Mobile license (VR device):

3 ID (VR device).

Confirmation required.

1	100-010	201-2a2	☒	×
---	---------	---------	-------------------------------------	---


Mobile license (VR device):

3 ID (VR device).

1	100-010			
2	<u>201-2a2</u>			



Possible error

- If you skip the Smart Center confirmation step the notification will appear
 - Confirm the license in Smart Center and proceed with next steps

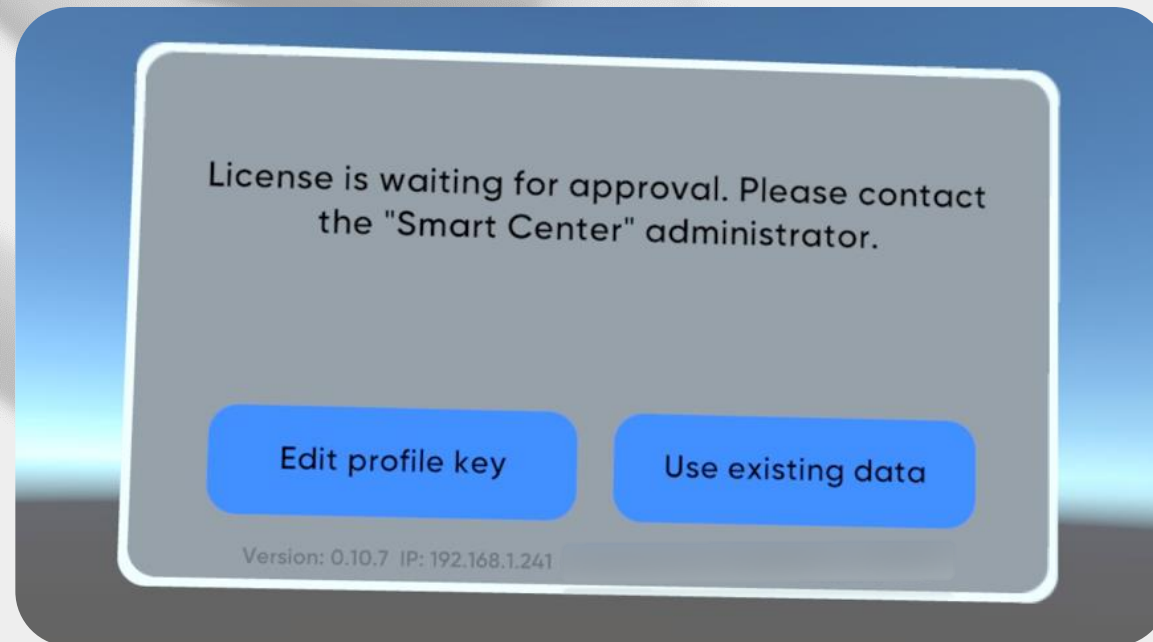



Finish XR-Clinic Mobile setup

- Click the “Edit profile key” button
- Enter your Profile key in the headset once again
- Download available cases from the Case Library

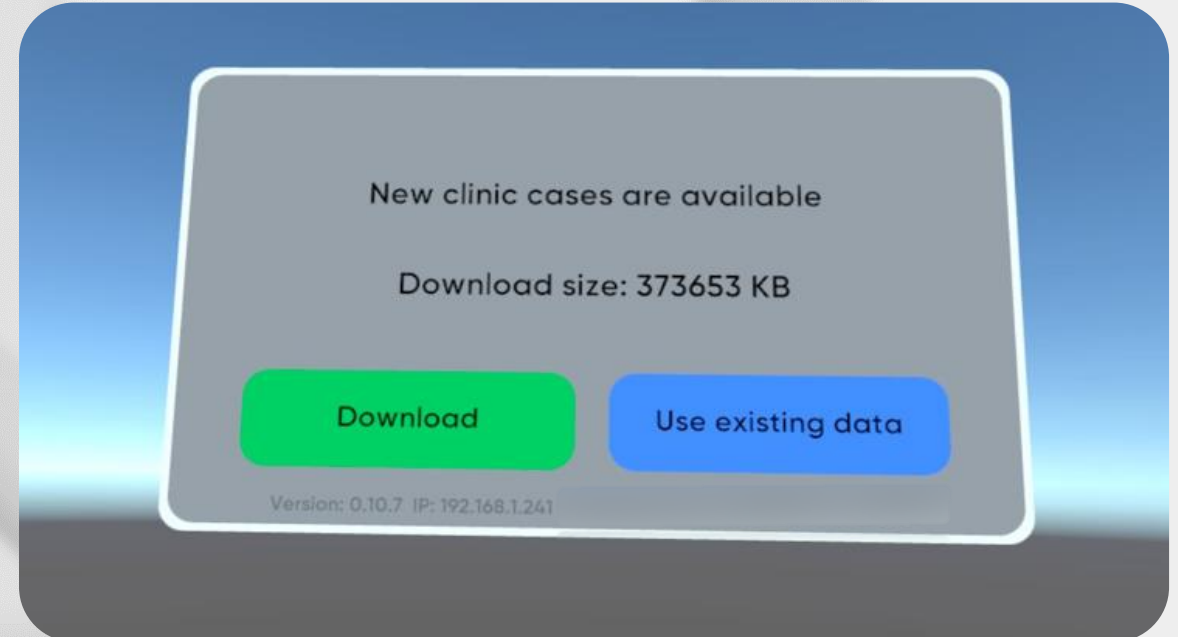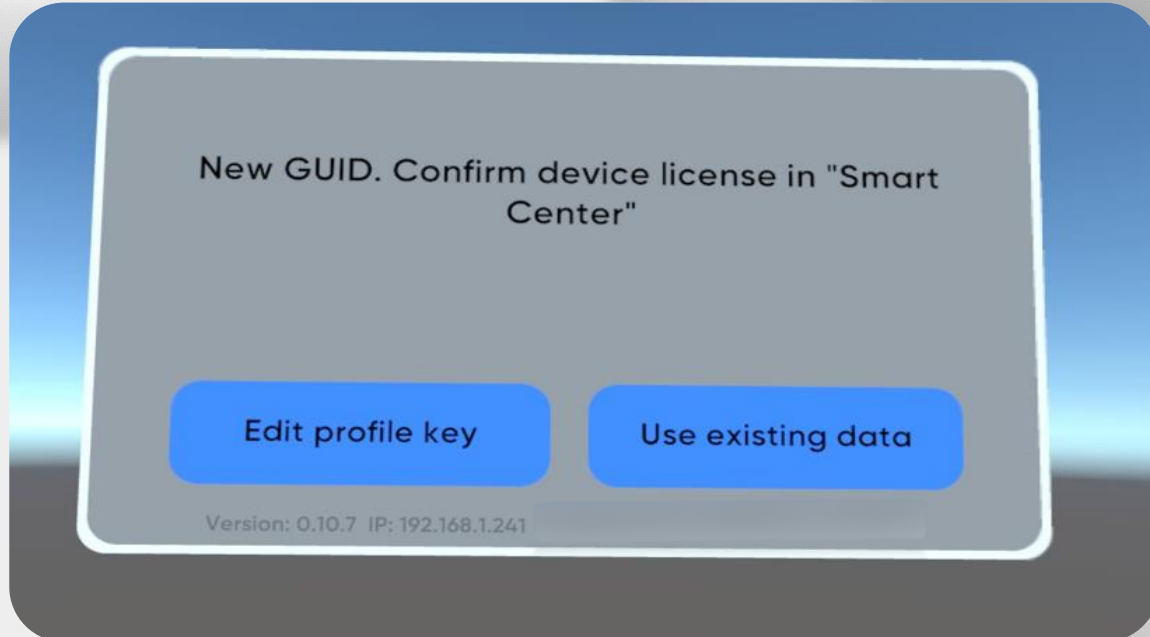

Finish the XR-Clinic Mobile setup

- Do not take your headset off while cases are downloading
- As soon as download is complete the XR-Clinic simulation will begin

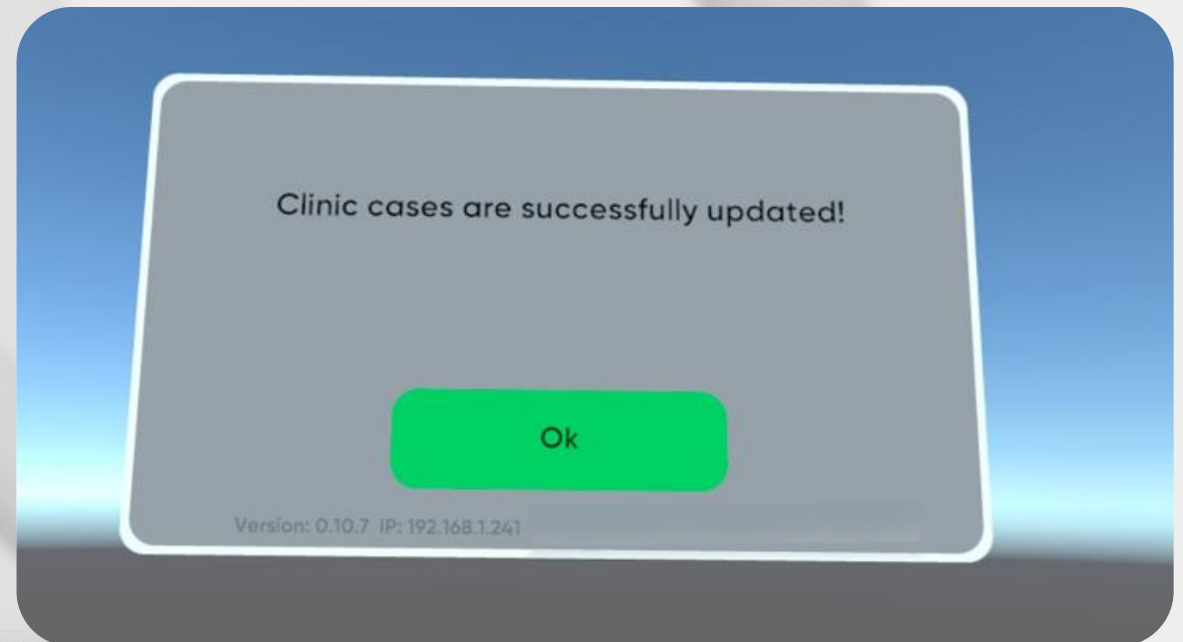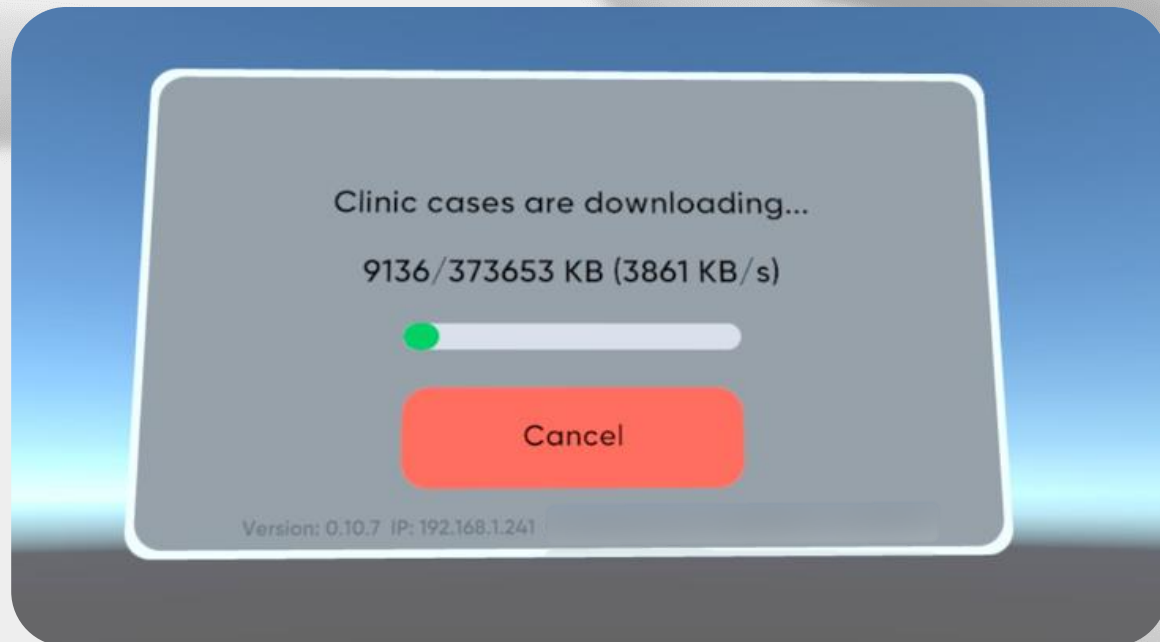

**Be
READY**

Technical support
service@mvredtech.com

Nasco
HEALTHCARE