

HOW TO UNCLOG IV CIRCULATION PUMP

1. Prepare the following items.



2. Connect them like the picture at the below.
Fill air into the syringe and inject it to the pump several times.



3. If the pump is still clogged, fill the syringe with water and inject it to the motor several times.



4. If this is not working, please try it 10 minutes later.



5. Connect all components as the picture at the below, circulate the pump for 1 minutes by pressing acceleration button.



6. Unplug the white tip from the bottle and vacuum all water inside of the pump. (It would be better to put the unplugged tip into a cup so that water from the pump can be collected.)

